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1 General instructions

1.1 Introduction

These operating instructions contain information on how to use the turning aid / turning and transfer aid safely and properly.

Read the operating instructions thoroughly prior to the use of the turning aid / turning and transfer aid and follow all instructions.

Keep all documents in a safe place.

1.2 Proper use

The turning aid / turning and transfer aid is used on a bath lift and utilised to rotate persons with reduced mobility into bathing position. The turning and transfer aid is additionally used to transfer persons with a more reduced mobility from the rim of the bathtub to the bath lift. Any other form of use is not permitted. The turning aid / turning and transfer aid may not be used by children. The turning aid / turning and transfer aid is not suitable for persons who are smaller than 146 cm or weigh less than 40 kg (BMI < 17). Locations in which the turning aid / turning and transfer aid is intended to be used include retirement and nursing homes, hospitals, rehabilitation centres and private households.

The **indication** is indicated for: damage to the lower and upper extremities with limited or partially reduced mobility and muscle strength / damage to vestibular functions with a tendency to fall (impaired balance) / damaged muscle functions with sufficient trunk stability.

Contraindication is indicated for: false sensations / severe disequilibrium / inability to sit.

1.3 Disposal

The product consists of recyclable materials.

Old appliances and packaging must be disposed of and recycled in accordance with the valid national regulations.

1.4 CE mark

The technical requirements for the product are set out in ISO 17966 (personal hygiene aids) and are met. The product complies with the medical products directive (EU) 2017/745.

1.5 Reuse

The product is suitable for reuse. The amount of reuse depends on the condition of the product in question.

Prior to reuse, follow the manufacturer's reconditioning and hygiene guidelines, and check the technical and functional safety of the product as well.

1.6 Service life

The product is designed for a service life of at least 3 years, provided that it is used in accordance with its intended use and that all cleaning and safety instructions are observed. Further use or inspection/repair of the product after this period is conducted at the discretion and responsibility of the operator.

2 Safety instructions

Do not overload the turning aid / turning and transfer aid (max. user weight: 140 kg).

Always put the weight on the centre of the rotary seat.

Hot water, sunlight or hot air can cause the surface of the product to become hot. Risk of burns!

Check the turning aid / turning and transfer aid for functionality prior to every use.

Do not use a damaged turning aid / turning and transfer aid.

In case of severely reduced mobility, use the turning aid / turning and transfer aid with the help of an assistant.

Serious incidents relating to the product must be reported to the manufacturer and the competent authority.

3 Description

The product is available in the following versions:

- Turning and transfer aid (Fig. 1A)
- Turning aid (Fig. 1B)
- B-protect (with antimicrobial coating) (without Fig.)

The turning aid / turning and transfer aid consists of a rotary seat (Fig. 1A/1B) with seat cover (Fig. 5) and a base (Fig. 2A/2B). The rotary seat is rotatably connected to the base. In the case of the turning and transfer aid, the rotary seat can be shifted, too.

The base has several moveable links which adapt to the bath lift such that the turning aid / turning and transfer aid does not tilt. Rubber pads on the moveable links prevent the turning aid / turning and transfer aid from slipping.

3.1 Technical data

Refer to "Technical data" at the end of the operating instructions.

3.2 Label

The label (Fig. 4) is attached to the bottom side of the turning aid / turning and transfer aid and contains the following information (exemplary illustration):

1 manufacturer; **2** max. user weight; **3** follow the operating instructions; **4** indoor use only; **5** date of manufacture (YYYY-MM-DD); **6** mark of conformity; **7** medical device; **8** UDI = Unique Device Identifier: (01) UDI-DI/GTIN, (10) batch / lot number, (11) date of manufacture (YYMMDD); **9** LOT = batch / lot number; **10** REF = item number; **11** TYP = model

4 Use

CAUTION Do not allow any oily or strongly soapy substances to get to the rubber pads. Otherwise, the turning aid / turning and transfer aid may slip from the bath lift.

CAUTION Do not reach underneath the rotary seat while rotating. Finger can be crushed.

CAUTION When using as a transfer aid, keep a still position and avoid excessive body movements. Otherwise, there is a risk of slipping.

CAUTION Do not use the turning aid / turning and transfer aid together with the seat cover of the bath lift. There is a risk of slipping.

4.1 Turning and transfer aid

1. Raise bath lift to entry height.
2. Place turning and transfer aid on seat section such that the rotary seat in the end position is in the centre of the seat section (1, Fig. 3A).
3. Push rotary seat towards the rim of the bathtub (2, Fig. 3A).
4. Sit down on the rotary seat.
5. With the rotary seat, move to the centre of the seat section (3, Fig. 3A).
6. Lift legs and swivel into the bathtub (4, Fig. 3A).
7. Lower bath lift.

To get off: First, slide to the rim of the bathtub with the rotary seat and, then, swivel legs out of the bathtub.

4.2 Turning aid

1. Raise bath lift to entry height.
2. Place turning aid in the centre of the seat section (1, Fig. 3B).
3. Sit down on the rotary seat.
4. Lift legs and swivel into the bathtub (2, Fig. 3B).
5. Lower bath lift.

5 Maintenance and care

5.1 Care

NOTICE Do not use any scouring agent. Otherwise, the plastic surfaces may be damaged.

The product can be cleaned with lukewarm water mixed with a mild detergent (without solvents and of a pH value of 5-9). Care for the plastic pieces with an off-the-shelf plastic cleaner.

The seat cover can be removed and washed at 40 °C in a washing machine. Do not dry the seat cover in a tumble drier.

If necessary, or when using the product for more than one person, the surface can be disinfected with an off-the-shelf disinfectant (max. 70 % alcohol) **after cleaning**. Depending on the disinfectant used, the disinfectant will be sprayed on or applied with a cloth. Follow the disinfectant or detergent manufacturer's instructions.

When adhering to the specified cleaning and safety instructions, the product is maintenance-free.

After cleaning, dry off the turning aid / turning and transfer aid with a clean cloth.

5.2 Removing the seat cover

1. Remove plug in the centre of the rotary seat (1, Fig. 5).
2. Pull seat cover slightly back at the rim until you can remove it (2, Fig. 5).

5.3 Attaching the seat cover

1. Place and align seat cover on the rotary seat.
2. Push edges of the seat cover into the recesses of the rotary seat (3, Fig. 5).
3. Fasten seat cover with the plug.

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© DIETZ GmbH, Karlsbad, Deutschland
DIETZ GmbH, Karlsbad, Deutschland.
© DIETZ GmbH, Karlsbad, Germany
DIETZ GmbH, Karlsbad, Germany.
© DIETZ GmbH, Karlsbad, Alemania
DIETZ GmbH,– Karlsbad, Alemania.