Neuromuskulärer Stimulator
Neuromuscular Stimulator

DoloBravo

Medizinprodukt / Medical Device

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Content

Please read the Operators Manual carefully before using the device
and pay attention to all references !

	Page
I. Neurostimulator DoloBravo	
Delivery & Accessories / Medical Device Reporting	29
Introduction - What is DoloBravo	30
Indication	31
Contraindications	31
Warnings	32
Advice on Application of Electrodes	33
Technical Data / Explanation of Symbols	34
Maintenance / Warranty	35
Setup Real Time Clock / Safety Adjustments	36
II. Display Elements of the DoloBravo	
Display and Operating Elements	37
Description of the Device	38
Battery Compartment with Lock-button	39
III. Application of the DoloBravo	
Preset Programs	40-41
Setting the Custom Programs	42-43
IV. Transcutaneous Electrical Nerve Stimulation (TENS)	
Gate-Control-Effect	44
Treatment Modes	44
Programs 01 - 12	45
Abbreviations	46
V. Control of Therapy	
Lock- / Unlock-Buttons / Statistics	47
Reading the Statistics	48-49
Trouble Shooting Guide	50
Placement of Electrodes	52-56
Waste Disposal / Recycling of Batteries / Literature / Copyright	57
Anatomy	58
Electromagnetic Compatibility	62-64

Delivery & Accessories

Included in the delivery:

REF 010E-001: 1 device **DoloBravo**, 2 lead wires, 4 self adhesive electrodes (40 x 40 mm), 4 batteries type AA and 1 operators manual

With regular average use, the expected service life of **DoloBravo** is 10 years. Service life of accessories is one year.

Accessories:

REF

011E-006: 1 set self adhesive electrodes 40x40 (4 pieces)

011E-007: 1 set self adhesive electrodes 40x80 (4 pieces)

011E-051: 1 set self adhesive electrodes 32mm Ø (4 pieces)

Spare parts:

REF

012E-013: 2 lead wires

052F-805: 4 batteries type AA

012E-045: 1 operators manual **DoloBravo**

012E-086: 1 carrying case

DoloBravo shall only be used with original accessories.

The batteries life is approximately 12 hours or more at 20 mA per channel. The storage time of the batteries is approximately 2 years. The service life (expiry date) of electrodes is printed on each package.

Max. environmental conditions for transportation: -25 to +70 degree centigrade, 0-93% atmospheric humidity. Recommended environmental conditions for storage: -25 to +40 degree centigrade, 0-93% atmospheric humidity. The device shall not to be stored in direct sunlight. Allow up to one hour for the device to return to normal operating temperature if stored in conditions below +5°C or above +40°C.

Environmental conditions for use: +5 to +40 degree centigrade, 0-93% atmospheric humidity, air pressure 700-1060 hPa. During use the surface temperature of the device can be 41°C or higher, but never exceeds 48°C.

Medical Device Reporting:

If there is a serious deterioration in the state of your health when using the medical device, report this to your distributor or to the manufacturer and to the competent authorities of your country.

Introduction – What is DoloBravo ?

DoloBravo is a digital electrical nerve-stimulator, which is made for pain relief. Doctors and patients are the intended operators for this device. This small battery operated unit provides a non-invasive, drug free method of controlling acute and principally long term intractable pain. It can also be used as an adjunctive treatment in the management of post surgical traumatic pain problems.

DoloBravo has been developed according to latest findings by MTR+ in consultation with medical professionals and patients. This device is a modern 2-channel stimulator, which enables 2 pairs of electrodes (= 4 electrodes) to work simultaneously.

DoloBravo is specially designed for highly efficient pain therapy (TENS) and can be used without any assistance. The mild electrical impulses are transmuted through the skin via surface electrodes to modify the body's pain perception.

DoloBravo contains 12 fixed programmes in four different modes.

The **continuous mode** blocks pain transfer very well.

The **modulating mode** blocks pain transfer and in addition delays the bodily habituation process to the electrical stimulation.

The **burst mode** is responsible for the increased release of body-own endorphins.

The **Dense-Disperse-Stimulation**, developed by Prof. Han, is a special kind of continuous mode. It includes a continuous change between 100 Hz and 2 Hz. This way, different analgesic effects can be achieved simultaneously.

DoloBravo has multi-phase programs, in which different modes alternate automatically. Therefore it is possible to achieve all of the above specified therapy goals with one single program. Additionally the three configurable programs enable the specialist to compile specially tailored programs for their own individual purpose. This characteristic of the **DoloBravo** offers each physician or physiotherapist, and also the experienced patient, the possibility of a focused co-ordinated treatment for the respective state of health. As these programs, after storing, can be changed at any time, for example to be altered according to the progress of therapy, an almost unrestricted variation range is available with this device to the user.

The small size of this neat but powerful unit makes transport easier. **DoloBravo** fits into any bag and can be used in holiday as well as at home.

The clip on the back of the unit allows to fasten the device at the clothes but the unit is intended to be supported by hand during normal use. The belt clip is for temporary relief only.

Indications

DoloBravo was designed for the following indications:

Nerve Stimulation:

- Increase of release of bodily endorphins (PRG 4, 7-12)
- General Pain Relief and Pain Therapy including but not limited to:
 - Cervical spine pain (PRG 1, 2, 5, 9, 10, 11)
 - Thoracic spine pain (PRG 1, 2, 5, 9, 10, 11)
 - Lumbar spine pain (PRG 1, 2, 5, 9, 10, 11)
 - Arthritis (PRG 3, 8, 12)
 - Circulation problems (PRG 1, 2, 3, 5, 6, 7)
 - Neuralgia (PRG 1, 6, 10, 11)
 - Myalgia (PRG 8, 9, 12)
 - Phantom pain (PRG 4, 8, 10, 11)
 - Fracture pain (PRG 1, 2, 5, 9, 10, 11)
 - Pelvic floor muscle pain (PRG 1, 3, 7)

Contraindications

DoloBravo may not be used in the following cases:

- By patients with electronic implants like pumps or a pace maker because functional disturbances can occur.
- By patients with myocardial damage (damage to the cardiac muscle) or arrhythmia (inconsistent heart beat).
- By patients with epilepsy.
- If severe local inflammation or thrombosis is present.
- By patients with mechanical ileus.
- By patients with febrile illnesses or infectious diseases.
- Do not use with a surgical high frequency apparatus because burn marks could occur underneath the electrodes.
- When using metal-containing contraceptive spirals.
 - If after an operation the healing process can be interrupted through a muscle contraction.
- After acute bleeding or bone fractures with a tendency to bleed, do not use in areas of bleeding or fractures.
- During menstruation or pregnancy.
- In case of desensitized skin with nerve damage.
- If stimulation is located close to osteosynthesis material (metal in the body).
- If patients cannot use the device properly, due to mental or physical limitations.

Warnings

Please pay unconditional attention to the following basic rules:

Do not stimulate transcerebral (via the head).

Do not stimulate via the Carotis-Sinus-Nerve (carotid artery).

Do not stimulate in the area of the Sinus knot (heart).

Do not stimulate on the Bulbous (eye).

Do not stimulate in the Larynx-Pharynx-Area (throat / mouth).

Do not stimulate in places with skin defects.

Do not position the electrodes directly on the spine.

Take special care with the use of the electric stimulation:

In presence of flammable anesthetic mixtures with air or with oxygen or nitrous oxide.

When operating machinery with a potential of danger.

While driving a vehicle.

DoloBravo is type BF equipment and may not be immersed in liquids. The **DoloBravo** is protected against debris larger than 12.5 mm in diameter from entering the cabinet and must be stored and used protected from moisture and water.

Burn marks could occur underneath the electrodes if used with surgical high frequency apparatus.

If device operation occurs close to short wave, microwave, cellular or other wireless equipment, instability of the values of the stimulator could occur causing a temporary reduction or loss in current. Move away from the source of interference and increase the current to continue the program.

Strong static discharge occurring to the device may interfere with its operation. Turn off the device and turn on again after a couple of seconds if device effected by a static discharge.

The application of the electrodes nearby the thorax can increase the risk of cardiac fibrillation.

Patients with metal and/or electronic implants may only stimulate with permission of an experienced doctor.

Stimulation in the areas of tumors is allowed only by instruction of the specialized physician.

The **DoloBravo** may not be operated if the battery compartment is not closed.

Before opening the battery compartment, make sure that there are no lead wires in the jacks at the head of the unit to eliminate any possibility of current flow.

The operator should not touch the battery contacts & patient simultaneously.

Never connect the device directly to a battery charger or to any other mains powered equipment.

If using rechargeable batteries be sure to use a CE approved battery charger.

If you have swallowed any part of the device or any of its accessories, please contact a doctor immediately.

TENS and muscle stimulation should be used during menstruation or by pregnant women as well as by children under the guidance of a doctor or therapist only.

Remove batteries and battery cover, drain water from device and allow (**DoloBravo**) to dry normally for 48 hours if accidentally submerged in water.

Please, keep this device stored out of the reach of children. Do not leave the lead wires in reach of children, since strangulation is possible due to the length of the lead wires.

Advice on Application of Electrodes

Make sure that **DoloBravo** is turned off (no indication visible on the LCD display).

Now connect the single poled ends of the lead wires with the electrodes.

The output plugs of the electrode lead wires will be plugged into the channels A or B of the **DoloBravo**.

Furthermore make sure, that all plugs fit completely into the electrode sockets as well as the device sockets.

Please don't use any oil, ointment or cream, since all these have an isolating effect.

Now place the electrodes according to the instructions of your doctor or physiotherapist.

In the case of the skin gel loosing its adhesion, you can't achieve more than 6 mA - please replace the electrodes. The same effect, however, also occurs when using a damaged lead wire.

Therefore, if the normal treatment is not possible by using new electrodes, the lead wire may be faulty.

Electrodes which were damaged due to an improper handling may not be used.

Electrodes with a damaged lead wire isolation (e.g. visible copper wire) may not be used.

Electrodes with damaged, missing or stained aluminium coating or carbon layer, also with insufficient adhesive gel may not be used.

Electrodes may not be altered (e.g. by means of scissors or other tools).

Do not pull the electrodes by force on the electrode lead wire.

Apply the electrodes on clean and healthy skin only.

In the case of strong bodily hair growth, the hair should be shortened with scissors.

Neither stretch the skin nor the electrodes when applying the electrodes.

To remove the electrodes from the skin, please lift the electrodes at the rim towards the middle.

Effective current densities of more than 2 mA/cm² require an increased attention of the user with regard to the intensity adjustment.

If you use self-adhesive electrodes, we recommend the use of our self-adhesive electrodes with an area of 40x40 mm.

All electrodes are made for use of one single person only.

Electrodes can contain amounts of nickel. Allergy sufferers should check this before using.

Should skin irritation occur whilst using the **DoloBravo**, please interrupt the application and inform your doctor - to exclude an allergic reaction.

Technical Data

Channels: Two isolated independent channels

Waveform: Asymmetrical, rectangular, biphasic with zero DC-current

Intensity: 0 - 90 mA into 500 ohm load,

70 mA maximum into 1000 ohm load

65 mA maximum into 1500 ohm load

(Indication only, actual intensity varies depending on condition of the electrodes)

Classification: Internally powered, constant current, continuous operation, type BF applied part, class IIa, IP 20

Frequency: 1 - 200 Hz

Pulse Width: 50 μ s - 450 μ s

Ramp up/down: 0,1 - 9,9 seconds

Time adjustment: 1 minute - 9 hours, 59 minutes

Power supply: 4 batteries 1,5 V type AA

Voltage max.: 180 V without load (80 V into 1000 ohm load)

Dimensions: L 141,6 mm x W 74 mm x H 40,4 mm

Weight: 250 g (incl. batteries)

Firmware version 3.5 or later

This device is quality controlled and fulfills all legal requirements for medical products.

Therefore this device carries the CE-sign „CE 0123“. The serial number is on the rear of the unit.

Manufacturer: MTR+ Vertriebs GmbH,
Kamenzer Damm 78, 12249 Berlin. Germany
www.mtrplus.com

Explanation of Symbols

	Attention Warning		Manufacturer
	Typ BF applied parts		Date of manufacture 2022 - 04
	Serial-Number of the product		CE-mark: conforms to essential requirements of the European standards
	Keep dry		Not suited for persons with cardiac pacemaker
	Batch Code		Read the instructions
	Medical Device		Unique Device Identification
	Temperature limitation (transport)		Keep away from sunlight
	Temperature limitation (storage)		Humidity limitation

Maintenance

DoloBravo has been designed to be maintenance free. Nevertheless, please adhere to the following advices:

If needed, please clean **DoloBravo** with a mild detergent, water and a damp cloth. Do not expose **DoloBravo** to extreme humidity and do not submerge under water.

You may clean the lead wires with a damp cloth. You may also from time to time treat the lead wires with talcum powder in order to prevent cracks and to improve the lead wires life.

A clean cloth, moistened with an alcohol free disinfection liquid can be used to clean the device. Please follow all instructions of the manufacturer of the liquid.

After termination of the therapy, **DoloBravo** can be re-used by other patients. Please clean the device with a mild disinfectant cleaner.

The **DoloBravo** does not require calibration. The **DoloBravo** is not repairable and contains no user serveable parts. No modification to the **DoloBravo** is allowed.

When changing the batteries, please make sure to insert the new batteries within 30 minutes after removal of the spent batteries to ensure the devices time and date is kept current. We recommend that you always have spare batteries with the device.

Warranty

MTR+ Vertriebs GmbH provides a 24 month warranty, commencing from the date of our delivery (invoice date), that in case of failure, the **DoloBravo** will be serviced or replaced free of charge. All returns must first be authorised by MTR+ Vertriebs GmbH in advance.

Excluded from this warranty are failures of the **DoloBravo** which result from excessive use, careless or improper handling as well as damage by force.

The warranty loses its lawful meaning if the producer MTR+ Vertriebs GmbH has not given its consent in writing prior to repairworks or servicing being carried out. Without this written consent the manufacturer will not return the device.

The manufacturer reserves the right of technical changes, which may serve to increase treatment success, the functionality of the device or its accessories.

Setup Real Time Clock

DoloBravo has a “Real Time Clock”, that shows year, month, day, hours and minutes to store detailed statistics of treatment data.

For setting time and date, please select any custom program 13-15 and press the **SET**-button for at least 5 seconds. You'll reach the configuration-mode. Now press **SET** once again for 10 seconds.

With the +/- button of channel A (left side) you can step forward to the following values

- hour (indicated by: H)
- minute (indicated by: MI)
- day (indicated by: dA)
- month (indicated by: MO)
- year (indicated by: Y)

The respective value is flashing and can be changed with the +/- buttons of channel B (right side). Finally you save all your settings by pressing the PRG-button once again.

Before starting the therapy, please set the clock once if necessary.

If **DoloBravo** has been without batteries for more than 30 minutes, the clock must be set again.

Safety Adjustments

1. The +buttons become inactive after one minute running time of a program to avoid an unintentional increase of intensity . This lock is switched off again, only by pressing the +button three-times repeatedly. Now an increasing of intensity is possible again for another minute.
2. If the electrical circuit should be interrupted during operation (> 6 mA), for example by losing an electrode or by a broken lead wire, the device stops stimulating immediately.
3. If any phase comes to its end, the intensity falls down to 8 mA automatically and must be reset by the user to the desired setting.

Display and Operating Elements

„ON/OFF“: By pressing the button once, the device will either be switched on or off.

„+/- buttons“: This button adjusts the intensity for the channels A and B. The device also has two independently controllable output channels A and B. When pressing the respective arrow buttons the intensity will either be increased or decreased in steps of 1 mA. 60 seconds after the last intensity regulation, a further increase may not be carried out due to an electronic barrier. Pressing the „+“ button three times, will release the lock and the intensity can be increased again. A reduction of the intensity is possible at any time.

Whilst configuring the programs 13 – 15 you may switch between the programmable parameters by pressing the left „+/-“ button (channel A), and through pressing the right „+/-“ button (channel B) the desired value/data for respective parameter can be set. In this way, the application time, the stimulation, the pause, rising pulse and falling pulse, as well as synchronous or asynchronous stimulation can be selected.

„PRG“: With this button the preset programs 01–12 or the three customizable programs 13–15 can be selected for individual use.

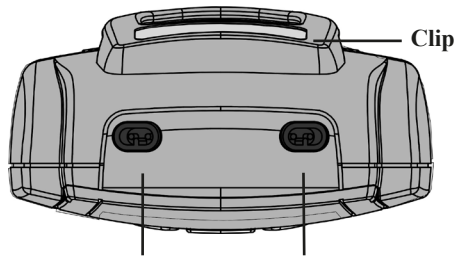
Pressing the PRG-button once at any time during a running program will reduce the mA on both channels to zero and pause the clock display with the actual treatment time. The clock continues to show the time of further treatment but it is frozen with the “:” symbol no longer flashing. In addition, this button stores the current adjustment in the programming mode and ends it. Then the device returns to the home screen of the selected program.

„SET“: With the SET-button you can select the different program phases. With each press of this button, the phase in progress is terminated and the next program phase is started. This applies to either the running of the programs 01-12 and also to the programming and the sequence of programs 13–15. Programs with one single phase will be finished by pressing the SET-button.

If you keep the SET-button pressed for 5 seconds whilst being in programs 13–15, the symbol of the clock is flashing and you will be able to start programming of your individual settings. Pressing SET once again for 10 seconds allows to setup the RTC.


Holding the „SET“ button for 5 seconds while the statistics are displayed, will display „dEL“. Holding „SET“ a second time for 5 seconds will delete the statistics, set all custom programs to default values and show the RTC setup screen.

Description of the Device

Battery Compartment with LOCK-button

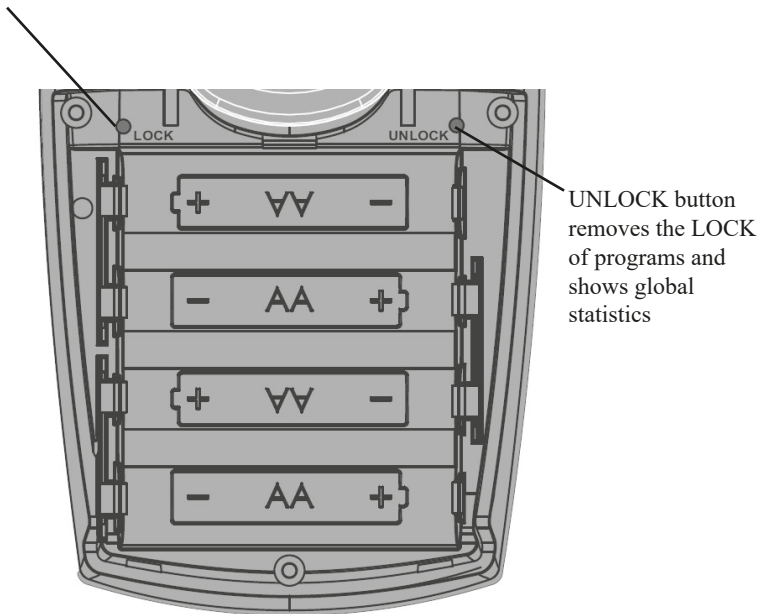DoloBravo allows the doctor or therapist to lock one program for the patient to use (s. p. 47). In this way the medical practitioner can be quite sure that the patient only uses the device with the correct program. The LOCK-button can be pressed with a pin of the lead wire or any other suited thin object. Pressing will fix the selected program so that different programs cannot be used anymore.

Arrangement of the batteries

Preset Programs 01–12

Preparations:

Make sure that **DoloBravo** is switched off. Cleanse the area of the skin which is to be treated with clean water. Following that, make sure that all plugs are completely connected to the respective electrode sockets as well as device sockets. Place the the adhesive electrodes according to the advices in pages 31-33 of this manual.

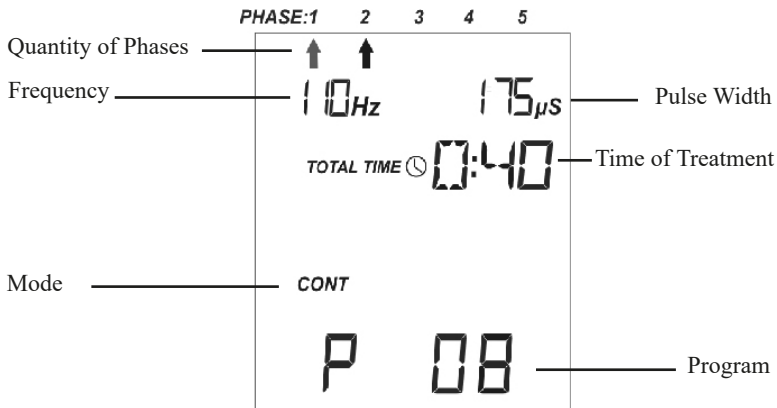
DoloBravo is operated by 4 batteries of type AA. Open the battery compartment at the back by pushing down the lid and insert the batteries. Please ensure the correct polarity in accordance with the markings. **DoloBravo** is now operational. If the battery voltage is too low, the battery-symbol in the LCD-display will flash. Please replace the battery at this time. Empty batteries constitute special waste and therefore have to be disposed of at an appropriate collection point. Never dispose of batteries via domestic or household waste.

Adjustment:

1. Press the button „ON/OFF” once.


DoloBravo is now switched on. The last used program appears on the LCD-display.

Parameters programmed for the first phase will be displayed as well as the quantity of all phases, the actual treatment mode and the total treatment time.

Preset Programs 01–12

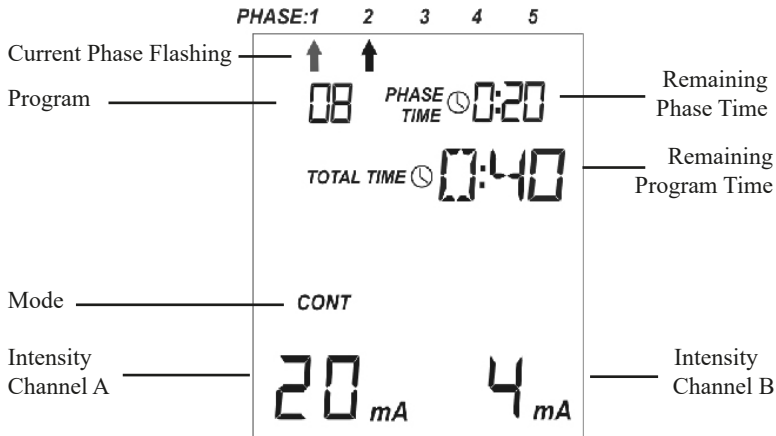2. By pressing the program button „PRG” you can choose from the programs 01 – 12 (pre-programmed).
3. This is followed by pressing the buttons „+/-” below the channel used.
Now the selected program runs automatically and the LCD-screen changes.

The current phase will be indicated by an flashing arrow and in the upper right corner you can see the remaining phase time. In the bottom line the intensity used in each channel appears. If the button „+” is pressed repeatedly the intensity of stimulation increases. If the button „-” is pressed repeatedly the intensity of stimulation is reduced. For the setting of the intensity it is vital that one feels the electric current but that discomfort is never incurred.

Please take into account that for safety reasons the intensity falls back to 8 mA at the beginning of each new phase and must be increased again.

Each phase end is indicated by 5 peep tones.

4. If you wish to interrupt the program, then press the button „PRG” once. The program will run again for the remaining program time when pressing the button „+”. In this case, the intensity must be reset to the desired level.
5. By pressing the „SET” button you end the current phase and move to the next. Programs with one single phase will be finished.
6. **DoloBravo** is switched off when pressing the button „On/Off” or automatically four minutes after the program is finished.

Setting the Custom Programs 13-15

1. Switch on **DoloBravo** with „**ON/OFF**” button.
2. Choose the desired program (13-15) with „**PRG**” button.
3. Press „**SET**” button for at least five seconds. **DoloBravo** changes to the programming mode and the upper watch, which indicates phase duration, flashes to show that this function can be altered.

In the following please use the left „+” button always then, if you want to store your choice and to change to the next step. The right „+/-” buttons are respectively used in order to alter the chosen and flashing parameter.

4. Select with the right „+/-”buttons the desired time for the first phase.
5. By pressing the left „+”button you will be able to store your choice and change to the next step. In the display the mode now flashes, which you adjust according to your wishes by pushing the right „+/-”buttons.

If you have chosen „**CONT**” or „**BURST**” you can determine frequency and after storage and change with the left +button respectively the pulse width by pushing the right „+/-” buttons.

If you have chosen „**WORK/REST**”- rhythm you can determine the following parameters one after the other by pushing the right „+/-”buttons.

- frequency (**Hz**)
- pulse width (**µS**)
- work time (**sec**)
- rest time (**sec** = recovery time)
- increase / decrease ramp up time of the pulse (**RU**)
- increase / decrease ramp down time of the pulse (**Rd**)
- synchronous (**SY**) or alternating (**AL**) output of the channels A and B. If you have selected “**SY**” you can program **delay-time (dL)** of 0 sec. up to 5 sec. from channel A to B.

If you have chosen „**MOD**” you can alter the following parameters one after the other.

- low limit of frequency (**Hz LO**)
- upper limit of frequency (**Hz HI**)
- low limit of pulse width (**µS LO**)
- upper limit of pulse width (**µS HI**)
- low limit of intensity (AdJ) - the display in the right corner below shows values from 0.5 to 1.0. This adjusts the intensity of current within the modulation time between 50% (=0.5) and 100% (=1.0) of the original set mA-value. The intensity, of a modulated program, can never increase above the 100% of the set value.
- modulation time (possible choice: 2 sec. up to 60 sec)

Setting the Custom Programs 13-15

6. Push the „SET” button to select the second phase of the program. You can program up to 5 phases of your own in each of the three configurable programs. If you want to reduce a configured program of 5 phases to less phases, you simply set the time of the first unwanted phase to zero.
7. The programming is completed by pressing the „PRG” button and the program is stored.

Selectable frequency and pulse width settings:

Continuous-Mode: 1 to 200 Hz, 50 to 450 μ S

Burst-Mode: 35 to 200 Hz, 50 to 200 μ S

Modulated-Mode: 1 to 200 Hz, 50 to 450 μ S

Work/Rest-Mode: 1 to 200 Hz, 50 to 450 μ S

It is not possible to program more than 300 μ S if the frequency is above 100 Hz.

The settings can be made as follows.

- Frequency: 1, 2, 3, 4, 5, 6, 7, 8, 9, 10, 12, 14, 15 Hz, and then increasing in 5 Hz steps until max. 200 Hz
- Pulse width; 50 to 450 μ S in steps of 10 microseconds
- Intensity: 50-100% in steps of 10%
- Modulation time: 2, 3, 4, 5, 6, 7, 8, 9, 10, 12, 15, 20, 30, 40, 50, 60 seconds
- Work time: 2-99 seconds in steps of 1 second
- Ramp up time: 0,1-9,9 seconds in steps of 1 second
- Ramp down time: 0,1-9,9 seconds in steps of 1 second
- Rest time: 2-99 seconds in steps of 1 second
- Delay: 0,0-5,0 seconds in steps of 0,1 second
- Session time: 1-599 minutes in steps of 1 minute

TENS

(Transcutaneous Electrical Nerve Stimulation)

The first treatment using electrical current was carried out around 2500 BC in Egypt. At that time they used electric sheatfish for the treatment of pain relief. The results obtained from this treatment were quite astounding bearing in mind that science concluded the physiological basis for this treatment in the 19th century. The Gate-Control Theory which was first published by Melzack and Wall in the 20th century formed the basis for the neurophysiological understanding and the first treatment of pain relief with the aid of high frequency TENS as is applied today.

TENS as a scientifically recognised method for pain relief therapy is a suitable aid for pain for many doctors. The stimulating electrical current impulses being produced by the nerve stimulator **DoloBravo** is transferred from the electrodes to the skins nerve fibres. The effect of pain relief begins at the start of the application and is normally effective for an amount of time after the stimulation period. This is achieved due to two different mechanisms:

Gate-Control-Effect:

The pain is transferred over the nerve fibres of the body to the brain which perceives the pain. Through stimulus of the nerves the **DoloBravo** nerve stimulator prevents further transmission of the pain signals to the brain. The pain sensation is therefore eased or inhibited.

Beta-endorphins-release:

TENS-therapy stimulates the synthesis and release of beta-endorphins. Beta-endorphins as bodily synthesised peptides, which have a similar effect as morphine for pain relief, but without the side effects that morphine carries (e.g. respiratory depression).

TENS is available in three different **treatment modes**.

1. **Continuous Tens (CONT):** Electrical impulses are exchanged between the electrodes in a continuous ongoing mode back and forth. They block the transmitting nerve bands. It is possible that the body gets used rather quickly to this traditional treatment. The effects can subside with prolonged treatment.
 - 1a A special kind of continuous mode is named **Dense-Disperse-Stimulation (CONT + BURST)** according to Prof. Han. The permanent alternation of frequency between 2 Hz and 100 Hz within a few seconds (2-4 secs.) raises the output of different analgesic substances.
2. **Modulated Tens (MOD):** By changing the parameters (frequency, pulse width and/or intensity) regularly and automatically, the sensation of an increasing and decreasing current is produced. Therefore the body will adapt distinctly delayed to the electrical impulses.
For example: Modulating time 10 seconds at a pulse range of 250 – 150 µs means: Start at 250 µs – exponential decrease (first fast then slow) down to 150 µs in five seconds, then increase (first slow then fast) back to 250 µs. This complete cycle lasts for 10 seconds.
3. **Burst (BST):** 9 impulses with a set frequency and pulse range, twice per second are emitted. This artificially produced frequency of 2 Hz is able to increase the production of the body's own endorphins and therefore relieve pain in a more natural way.

Programs

PRG	Phase 1			Time min	Phase 2			Time min
	Modus	Hz	μ S		Modus	Hz	μ S	
P 01	MOD	70/110	200/100	30				
P 02	MOD	100 Hz, Intensity mod. 100%-60%	175	30				
P 03	MOD	80/10	150/300	30				
P 04	MOD	2/5	250	30				
P 05	CONT	110	175	30				
P 06	CONT	135	100	30				
P 07	BURST	2 x 9 Pulses each second (80 Hz)	150	30				
P 08	CONT	110	175	20	BURST	2 x 9 Pulses each second (150 Hz)	200	20
P 09	MOD	70/110	200/100	20	BURST	2 x 9 Pulses each second (80 Hz)	150	20
P 10	HAN	100 Hz / 200 μ s changing continuously with 2 Hz / 300 μ s in 3:3-seconds-rhythm						30
P 11	HAN	100 Hz / 200 μ s changing continuously with 2 Hz / 300 μ s in 2:4-seconds-rhythm						30
P 12	HAN	100 Hz / 200 μ s changing continuously with 2 Hz / 300 μ s in 4:2-seconds-rhythm						30

Abbreviations

AdJ:	Adjustment of intensity
AL:	Alternate function of the channels A and B
BURST:	The burst mode delivers 2 blocks of 9 impulses each per second
CONT:	In the continuous mode a permanent unchanged current is used
dA:	Day
dEL	Delete
dL:	Delay between channels A and B
HAN:	Change of frequency and pulse width in a 3:3 seconds rhythm (indicated as CONT + BURST)
HI:	Upper value (high)
Hz:	Hertz is the measurement unit for the frequency of impulses per second.
LO:	Lower value
mA:	Milli-amp is the measurement unit for the electrical current intensity.
MOD:	In the modulated mode the frequency and/or the pulse width and/or the intensity changes in a predefined rhythm automatically.
Rd:	Ramp down
RU:	Ramp up
SEC:	Seconds
SY:	Synchronous function of the channels A and B
TI:	Time
TM:	Time of modulation-interval
W/R:	Work/Rest mode
μS:	Microsecond is the measurement unit for the length of every single impulse.

LOCK-/UNLOCK Buttons

The **DoloBravo** offers the ability to lock one program in place so that other programs or settings cannot be selected or changed. This allows the doctor or therapist to ensure that the patient uses only the program that has been prescribed.

The LOCK button in the battery compartment (s. p. 39) can be used by means of the pin of the electrode lead wire. To do this, please disconnect all lead wires from the device, switch on the device and open the battery compartment. After pressing LOCK you hear a single beep and the selected program is locked. In this case, you cannot use any other program anymore until UNLOCK is pressed.

Pressing the UNLOCK button in the battery compartment (s. p. 39) releases the program lock. A second press shows the global statistics on the display. If the unit was not locked, these statistics will be displayed on the first press.

Statistics

The therapist can read information about the compliance of the patient on the display at any time he wants. This way it is possible to evaluate whether the device was used according to the instructions of the doctor (quantity, intensity, etc.)

DoloBravo offers the chance to get a quick survey about the stored data of all treatment-sessions. The global statistics display the following averages:

- number of treatments per day
- number of days in use
- most used program
- total time of use
- intensity per channel

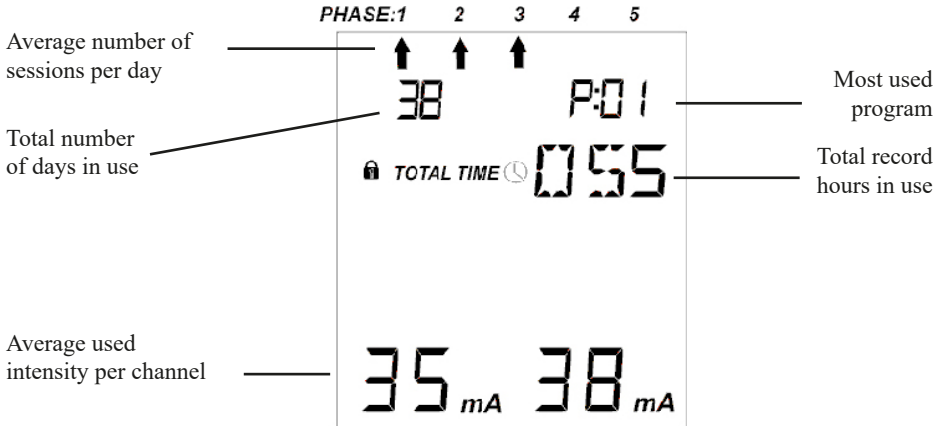
If more detailed information is required, **DoloBravo** allows the therapist to have a closer look at each single session of the last 60 treatment-days. You can see:

- number and date of the treatment-day
- number of sessions on this day
- time of session-start
- program used in this session
- total-time of the session
- intensity used for each channel in this session


Reading the Statistics

If you want to read the memory, switch on the device and press the UNLOCK-button in the battery compartment twice if the unit was locked in one program or once if it was not locked. The global statistics can be viewed on screen now.

Global Statistics:


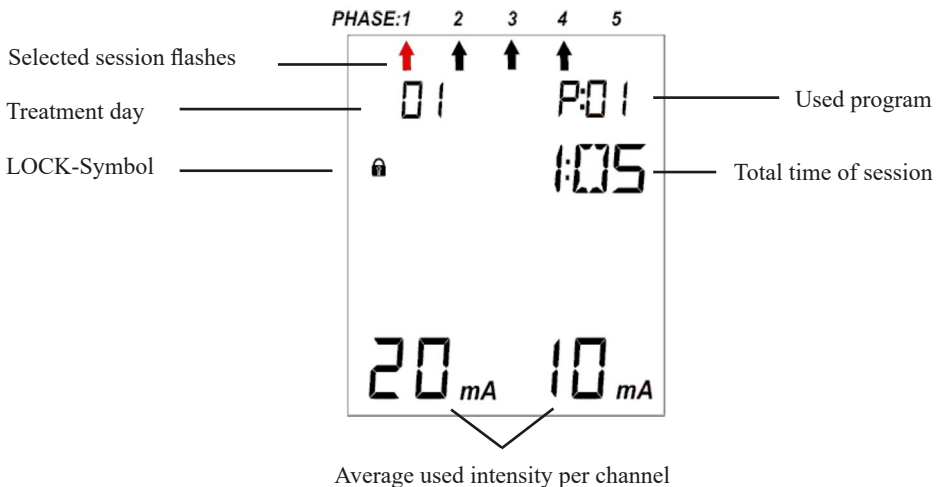
To get more detailed information press the + button of the left channel (A). You will see the data of the first session of the first day.

Reading the Statistics

The next sessions of the selected day are displayed - one after the other - by pressing the „SET“-button. Each next day can be obtained by pressing the + button of channel A again. If this button is held for at least 3 seconds, the device returns to global statistics.

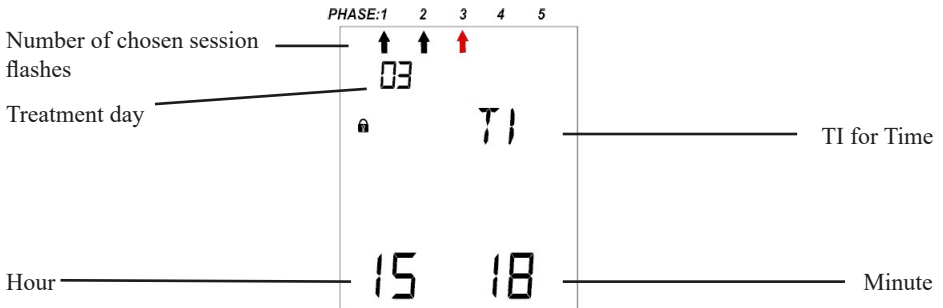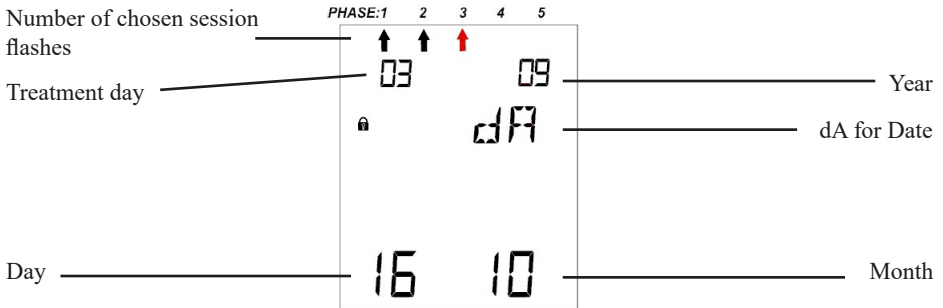If you require information about time and date of one single session please press any of the right +/--buttons (channel B). With these buttons you can scroll between session data, date and time.

The data stored in memory is retained after viewing, so that therapy-control is possible anytime. A maximum of 60 days of data can be stored with 5 sessions per day. When 60 days of data are reached, consecutive days will overwrite the earliest days. The day number will continue increasing (61, 62, ...) and will be displayed when the unit is unlocked until the **DoloBravo** is reset.

If you press the PRG button for three seconds, you return to the home screen of the selected program. The device will not be locked. If the therapy shall continue with only one program available, the LOCK-function must be reactivated after reading (s. p. 47).

The memory should be erased completely before a new person is using the device. After pressing the UNLOCK-button you have to press SET twice for 5 seconds each. All individual settings in programs 13-15 will be reset as well. The **DoloBravo** is set to its default state and all statistics are deleted.

Trouble Shooting Guide

Fault	Possible Cause	Solution
Trip off - Electrode symbol flashing, current won't increase above 8 mA	Electrode lost adhesion to skin.	Check electrodes and replace if necessary.
	Electrode too dry and loses adhesion to skin.	Replace electrode.
	Lead wire not connected properly.	Check lead wire connections and ensure they are fully inserted.
	Lead wire is faulty.	Replace lead wire.
	Device is damaged.	Return device to manufacturer for repair.
Device does not power on.	Flat battery or no batteries installed in device.	Check / Replace batteries.
	Battery installed in wrong orientation.	Remove batteries and install according to diagrams in battery compartment.
	Battery connector damaged or flattened.	Remove batteries, check metal parts in battery compartment for damage.
	Device is damaged.	Return to manufacturer for repair.
Device powers off during use.	Batteries almost flat.	Check battery icon during use and replace batteries if low.
	Device is damaged.	Return to manufacturer for repair.
Current output is unstable	Device is in use near a device emitting strong radio or magnetic interference.	Move at least four meters away from the offending device and resume treatment.
Treatment is painful or uncomfortable when using skin electrodes.	Patients skin has cuts or abrasions in the area of electrode placement.	Discontinue use until skin is fully healed.
Skin becomes red after treatment.	Device is being used for periods which are too long for the patient.	Use device for a shorter treatment duration.
Treatment is uncomfortable, tingling sensations during use.	Intensity too high for required treatment.	Reduce intensity setting until treatment is comfortable. Note: Tingling sensations are normal and should be felt.

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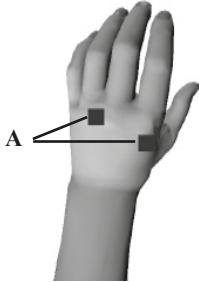
In most cases it is very easy to find the suitable points for the placing of the electrodes. It is frequently meaningful to take the hurting place to the middle between the two electrodes. We nevertheless recommend to coordinate the points of electrode-placement with the doctor or physiotherapists if possible. The distance between the electrodes shouldn't be much more then 20 cm. The polarity of the electrodes is without any significance in TENS therapy.

You see some examples of possible placement-points on the following pages.


Please follow in each case absolutely the references on page 31-33 of this manual.

While stimulating, please keep this device in reach, to be able to finish the treatment at once, if current becomes uncomfortable.

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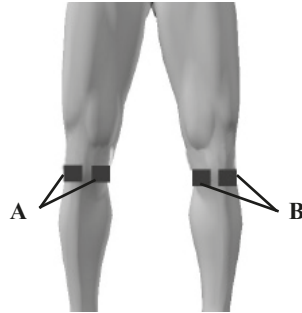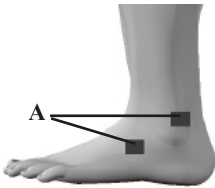
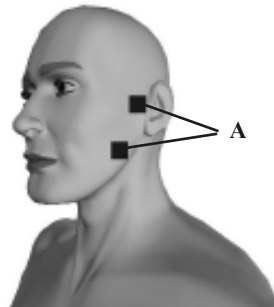
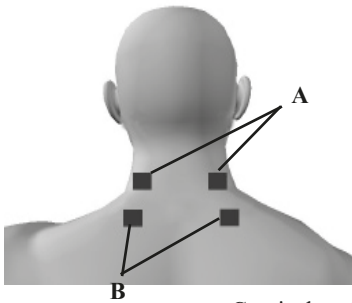
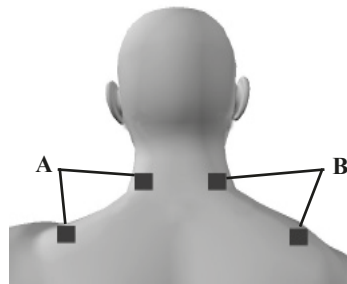
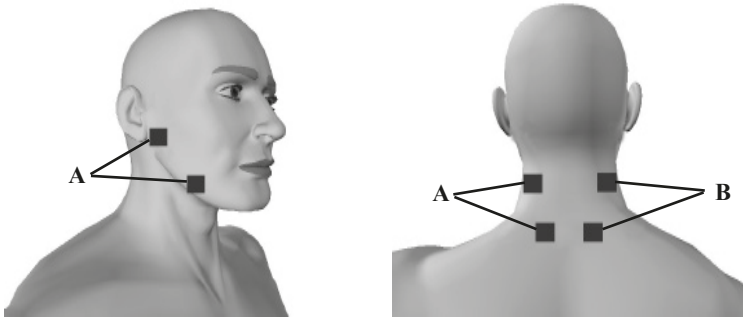
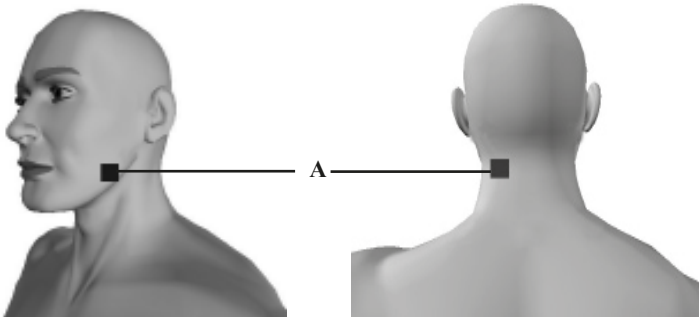
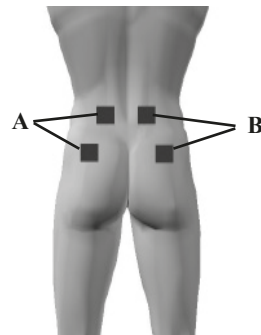
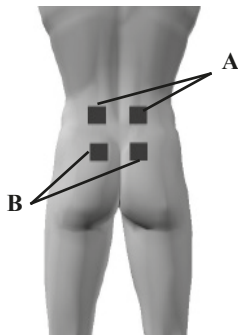
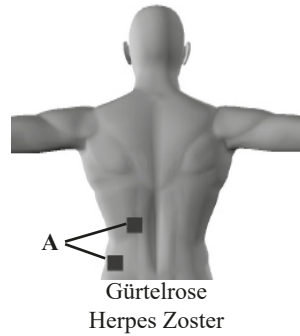
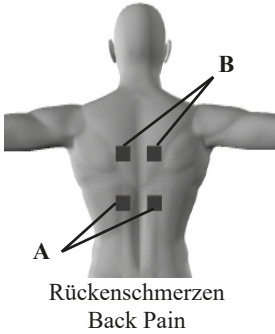
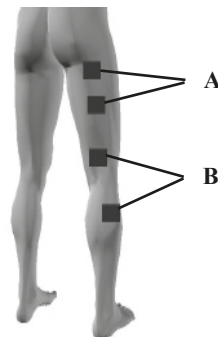
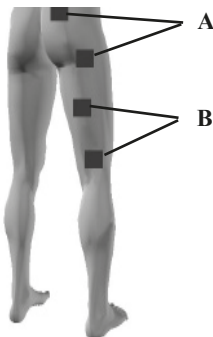
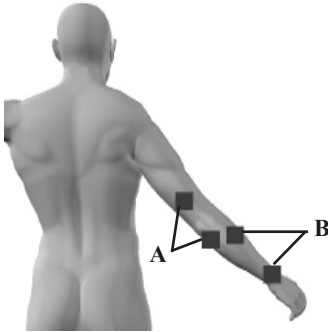
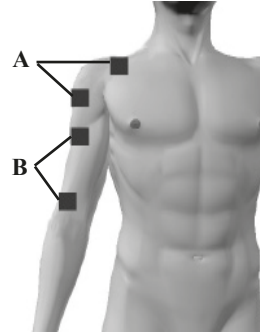
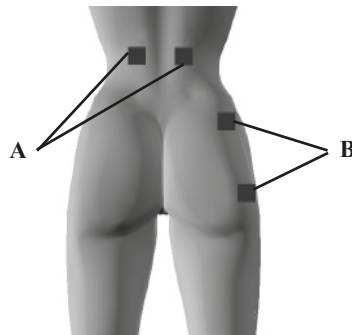
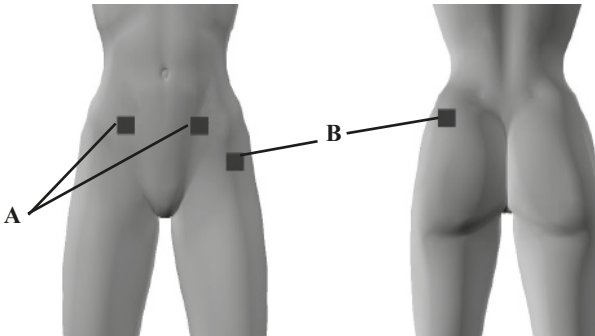
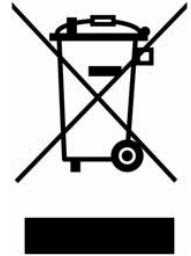
In accordance with European regulations [1], used electrical and electronic equipment may not be discarded in household or unsorted municipal waste. Use separate collection facilities. Contact your local government or council for information regarding the collection systems available. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the ground water and get into the food chain damaging the health and well being of people and animals.

[1] European Community directive 2012/19/EU on waste electrical and electronic equipment.

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The batteries should be replaced when they are drained or stop functioning. The batteries supplied with the **DoloBravo** are recyclable. Remove the batteries from the unit and follow your local recycling guidelines.

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A list of literature regarding the topics of this manual can be found on our website „www.mtrplus.com“.

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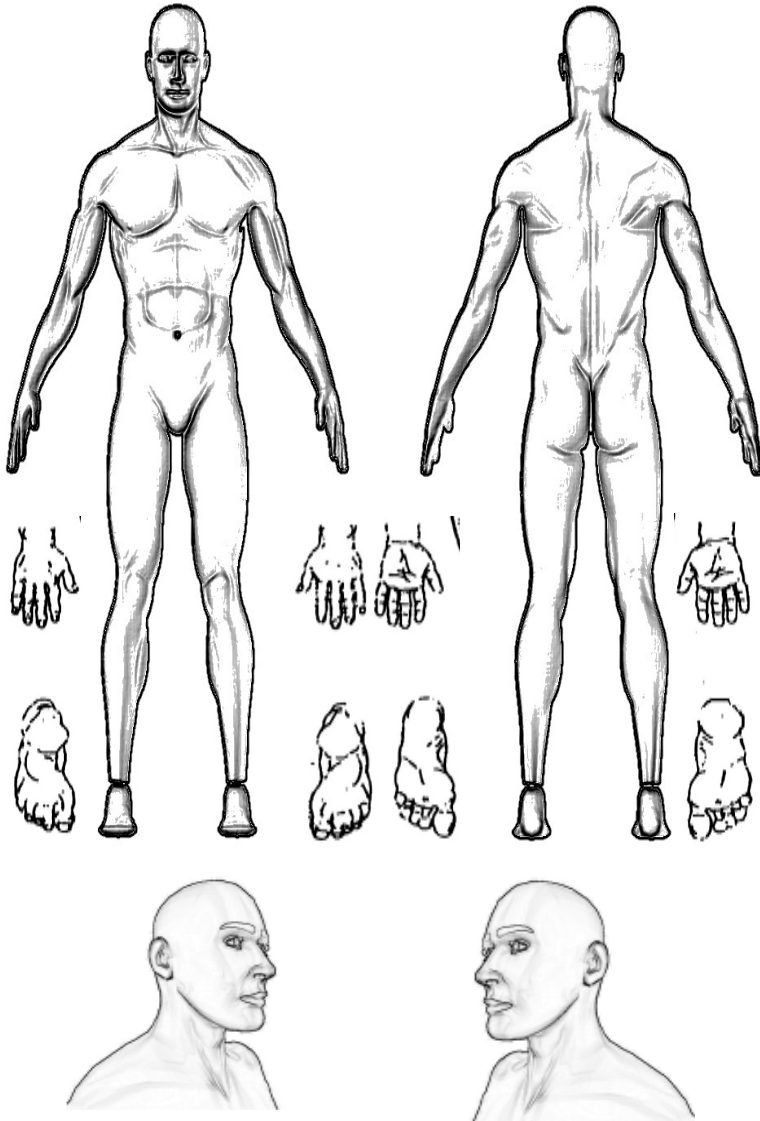
Reserved for changes.

Printing or copying this manual - also in an abridged version - is forbidden and can only be allowed with a written approval of “MTR+ Vertriebs GmbH”.

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Electromagnetic Compatibility

If the device is used according to the instructions given in this manual, it corresponds to the safety requirements for medical electrical systems IEC 60601-1. **DoloBravo** was checked and is in accordance with the limits set for medical products in the standard IEC 60601-1-2:2020. These limits serve to the safety of the user regarding electromagnetic disturbances. The **DoloBravo** is a Class B device and is suitable for home healthcare environments as well as typical clinical environments.

Guidance and manufacturer's declaration - electromagnetic emissions		
The DoloBravo unit is intended for use in the electromagnetic environment specified below. The customer or the user of the DoloBravo should ensure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The DoloBravo uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The DoloBravo is suitable for use in all establishments, including domestic and medical establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not Applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not Applicable	

Please adhere to these warnings when using the device:

Don't use **DoloBravo** near active HF surgical equipment or RF shielded rooms used for magnetic resonance imaging, where the intensity of EM disturbances is high.

Use of this equipment adjacent to, or placement on top of other equipment should be avoided in case the output of the **DoloBravo** is effected. If this is necessary, Both the **DoloBravo** and the other equipment should be monitored to verify that they are both operating normally.

Use of accessories and lead wires other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used not closer than 30 cm (12 inches) to any part of the **DoloBravo**, including lead wires specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

The performance of the **DoloBravo** can be effected if exposed to frequencies of 85 Mhz to 340 Mhz. If the **DoloBravo** is exposed to RF devices operating at these frequencies, the output of the current may be reduced. If this occurs, move to a position further away from the RF equipment, the output will stabilize when the **DoloBravo** is moved to a suitable position.

Electromagnetic Compatibility

Guidance and manufacturer's declaration - electromagnetic Immunity			
The DoloBravo is intended for use in the electromagnetic environment specified below. The customer or the user of the DoloBravo should assure that it is used in such an environment			
Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %
Electrical fast transient/burst IEC 61000-4-4	Power supply lines: ±2 kV input/output lines: ±1 kV	Not Applicable	DoloBravo is powered by batteries
Surge IEC 61000-4-5	line(s) to line(s): ±1 kV. line(s) to earth: ±2 kV. 100 kHz repetition frequency	Not Applicable	DoloBravo is powered by batteries
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° phases 0% U_T for 0.5 cycle And at single phase 0° 0% U_T for 1 cycle 70% U_T for 25/30 cycles 0% U_T for 250 cycles 0% U_T for 300 cycles	Not Applicable	DoloBravo is powered by batteries
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	30 A/m 50 Hz / 60 Hz	30 A/m 50 Hz / 60 Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC61000-4-6	150 KHz to 80 MHz: 3 Vrms 6 Vrms (in ISM and amateur radio bands) 80 % AM at 1 kHz	150 KHz to 80 MHz: 3 Vrms 6 Vrms (in ISM and amateur radio bands) 80 % AM at 1 kHz	Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	
NOTE 1: U_T is the a.c. mains voltage prior to application of the test level. NOTE 2: At 80 MHz the higher frequency range applies. NOTE 3: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

Electromagnetic Compatibility

Guidance and manufacturer's declaration - electromagnetic Immunity

The **DoloBravo** is intended for use in the electromagnetic environment specified below. The customer or the user of the **DoloBravo** should assure that it is used in such an environment

	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Modulation (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	385	380 – 390	TETRA 400	Pulse modulation 18 Hz	1,8	0,3	27
	450	380 – 390	GMRS 460, FRS 460	FM c) ± 5 kHz deviation 1 kHz sine	2	0,3	28
	710	704 – 787	LTE Band 13, 17	Pulse modulation 217 Hz	0,2	0,3	9
	745						
	780						
	810	800 – 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0,3	28
	870						
	930						
	1720	1 700 – 1 990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0,3	28
	1845						
	1970						
	2450	2 400 – 2 570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0,3	28
	5240	5 100 – 5 800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0,2	0,3	9
	5240						
	5785						

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