## LED lampe Rollator. TOPRO Rollatorer

### LED lamp Rollator. TOPRO Rollators

### Led Lampe Rollator. TOPRO Rollators


Ryggbøyle  
Back support  
Rückengürt.

Troja Classic

Troja 2G.

Olympos

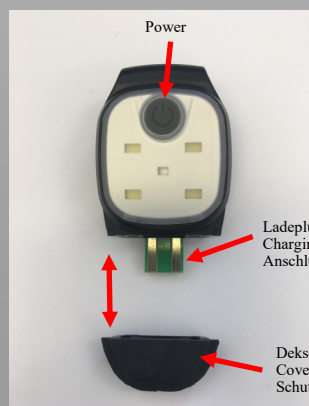**Innhold:** 1 x 104277 LED lampe Rollator  
1 x 815260M1 Monteringsanvisning

**Content:** 1 x 104277 LED Lamp Rollator  
1 x 815260M1 Mounting instructions

**Inhalt:** 1 x 104277 LED Lampe Rollator  
1 x 815260M1 Montageanweisung

Montering

Mounting

Montage

## English

"The LED lamp is equipped with an integrated rechargeable battery and can be recharged through the USB port of your computer. Since this product uses a rechargeable lithium-ion polymer battery, it must be recharged at least once of three months in order to prolong the life of the battery and protect the battery from dam-

**Use:** Press the power button to switch between the three light levels, red flash and off. 1st push: High level. 2nd push: Medium level. 3rd push: Low level. 4th push: Red flash. 5th push: Off

**Charging:** Remove the protective cover of the USB connector. Choose your preferred USB charger. Charging time: approx. 1 hour, depending on the charger. Red light when charging, green light when fully charged. **Please note:** If the charge light is not on, turn the USB connector.

**Warning:** The LED light is not a toy. Only to be used as intended.

**LED lampe rollator.  
TOPRO Rollatorer**
*LED lamp Rollator.  
TOPRO Rollators*
**Led Lampe Rollator.  
TOPRO Rollators**

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