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User Instructions

Bathing Board Deluxe with Soft Swivel Seat

Item No. 78061 000

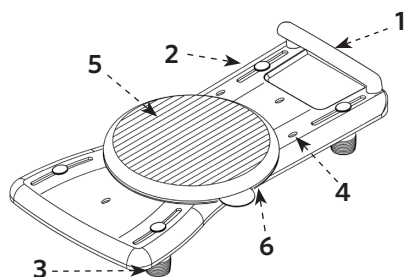
Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

The bathing board Deluxe with soft swivel seat offers people with difficulty standing up or standing alone but with the ability to sit up on their own support during bathing. The swiveling seat can be moved in 90° steps all the way around (360°) and simplifies getting in and out of the bath tub. Once the soft swivel seat is locked into place the seat glides along the board towards the middle of the tub or back to the edge. A handle ensures enough safety and stability during a transfer. The bathing board is punched in order to allow water to flow through it and the non-slip pads on the bottom side prevent damage to your bath tub.

Parts:

1. Handle
2. Bathing Board
3. Support Legs
4. Holes
5. Soft Swivel Seat
6. ID Plate


⚠ Safety Indications ⚠

- People weighing more than 150 kg must not use the bathing board.
- Please check the stability of the board, the handle and the legs prior to each use.
- Only use the bathing board when it can be securely installed.
- Please ensure that when the support legs are adjusted to the width of the bath tub that these are placed correctly against the side of the tub in order to ensure the stability of the board. The support legs should not be too wide or too tight against the side of the tub.
- Make sure that the soft swivel seat is locked when entering and exiting the tub as well as during bathing. Please also ensure that the seat locks after each turn of the seat.
- The bathing board is suitable for your bath tub when it does not extend more than 2 cm over the side of the tub and is not more than 0.5 cm from the outer side of the tub.
- Do not swing back and forth on the bathing board. This type of movement could cause the board to fall off the edges of the tub.
- Do not lean too far forwards or backwards. The bathing board could fall off the edges of the tub if a lot of weight is placed on one side of the board.
- Children may not use or play with the board alone due to the danger of falling off the edges of the tub.
- Please note that fingers can get caught in the openings and under the lower side of the seat!

Contents

The bathing board Deluxe is delivered with a soft swivel seat »A«, handle »D« with 2 screws »E«, 4 support legs »B« with 4 screws »C« and an instruction manual.

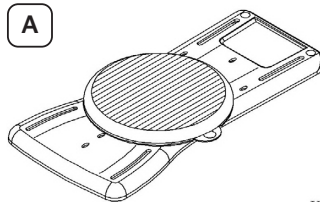A

X1

B

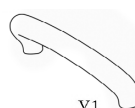
X4

C

X4

D

X1

E

X2

Assembly

Handle

In order to assemble the handle »D« please place it in the intended assembly openings so that it is facing you (away from the wall) and tighten the screws »E«.

Support Legs

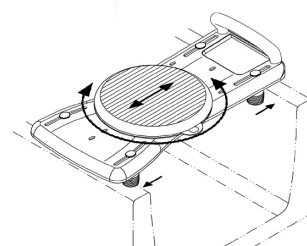
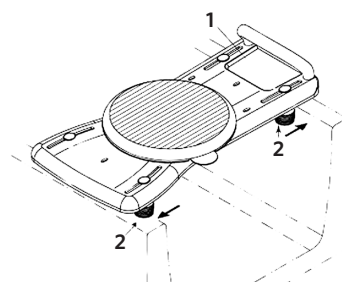
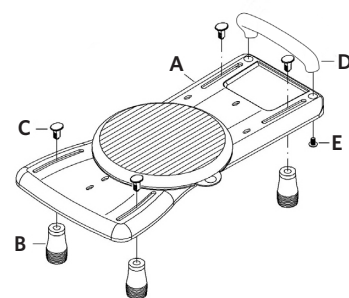
Screw on the four legs »B« with the screws »C« so tightly that they are secure but can still be moved. The legs will be tightened after determining the correct width in the tub. Also see »Assembly of the Bathing Board in the Tub«.

Assembly of the Bathing Board in the Tub

In order to set the correct width of the board please place the board on the edges of the tub. The wide side of the board is facing you and the handle is on the opposite/wall side of the tub (1).

Now you can adjust the legs to the desired width (between 56 and 68 cm)(2). Please tighten the screws in order to secure the correct width.

The support legs of the bathing board are shaped to provide excellent support – when installed correctly and at the correct width.




Use

The wide side simplifies entering the bath tub and the narrow side allows easy intimate hygiene. The handle on the narrow side supports entering and exiting the tub and provides support during use.

Soft Swivel Seat

The soft swivel seat can be moved in 90° steps all the way around (360°) and simplifies getting in and out of the bath tub. In addition, the seat can be moved horizontally 10 cm in order to move into the desired position.

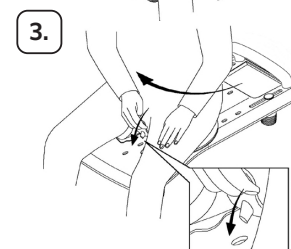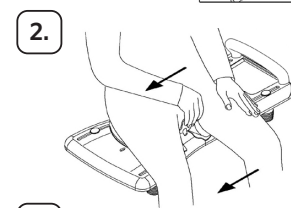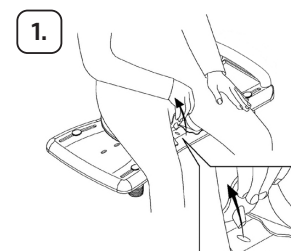1. Have a seat on the soft swivel seat and pull up the lever. By pulling handle the seat can be swiveled. Swivel the seat carefully and put one foot after the other into the tub. Hold onto the handle or have someone assist you if you feel unsafe when using the seat.
2. You can lock the seat in 4 positions. Place the lever into one of the according positions.
3. Please make sure the lever is securely placed into the according position. This prevents undesired swiveling and movement while bathing.
4. You can glide back and forth on the board in order to move into an ideal position for bathing.

Cleaning/Disinfection

The bath board should only be used on intact skin. A cleaning agent that is gentler to material should be used for manual cleaning. Only DGHM-/VAH-listed material-friendly surface disinfectant should be used. Please see the relevant manufacturer's instructions for a selection of suitable cleaning agents and disinfectants and their use.

Storage

The bath board should not be stored outdoors.

Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

Technical Data

- Dimensions (L x W x H): 72.8 x 35.4 x 7 cm
- Weight: 3.75 kg
- Diameter of Soft Swivel Seat: 32.1 cm
- Suitable for tubs with a width of: 56–68 cm
- Max. Load: 150 kg

This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at info@ruska.de as well as the competent authority, the Medicines & Healthcare products Regulatory Agency MHRA, www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency.

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Fabrikant: Ludwig Bertram GmbH, Im Torfstich 7, 30916 Isernhagen, Duitsland, www.russka.de