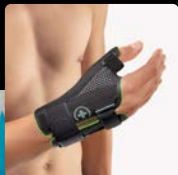
# BORT

## Daumenschiene Sport

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Sport

## Sprachen Languages

<b>DE</b>	deutsch	Gebrauchsanweisung .....	03
<b>EN</b>	english	Instructions for use .....	07
<b>FR</b>	français	Mode d'emploi .....	10
<b>ES</b>	español	Instrucciones de uso .....	13
<b>IT</b>	italiano	Istruzioni per l'uso .....	16
<b>NL</b>	nederlands	Gebruiksaanwijzing .....	19
<b>CS</b>	český	Návod k použití .....	22
<b>ET</b>	eesti	Kasutusjuhend .....	25
<b>PL</b>	polski	Instrukcja użytkowania .....	28
<b>RO</b>	româneșc	Instrucțiunile de utilizare .....	32

PDF: [ga.bort.com](http://ga.bort.com)

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Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

### **Intended purpose**

This medical device is a brace for the immobilisation of the thumb saddle joint and/or metacarpophalangeal joint.

### **Indications**

Post-operative, post-traumatic or degenerative irritations in the thumb saddle joint region, e.g. arthrosis of the CMC-I joint (thumb saddle joint), instability of the CMC-I joint, post-operatively following arthroplasty of the CMC-I joint, instability of the CMC-I joint.

### **Contraindications**

Lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated.

### **Application risks/Important notes** ⚠

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- remove it during longer periods of rest, e.g. when sleeping
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds

- do not use in case of intolerance of one of the materials used
- no re-use. This medical aid is intended for treating one patient only
- whilst wearing the brace, please neither use any creams nor ointments on or around it as they can destroy the material

### **Fitting instructions**

Open all the Velcro fasteners and slip the hand affected into the brace. Guide the thumb through the intended opening. Close the circumferential fastener on the thumb first, then the narrow fastener on the wrist. Finally, close the elastic fastener on the lower arm. Avoid congestion in the hand area due to closing the fastening straps too tightly.

### **Removal**

To remove the aid, open the harnesses and then take the aid off. Close all the Velcro fasteners in order to avoid damaging other items, e.g. to further clothing.

### **Material composition**

Polyamide (PA), polyester (PES), cotton (CO), elasthane (EL)

The sewn in textile label provides the precise material composition.

### **Cleaning information**

 Delicate wash 30 °C  Do not bleach  Do not dry in a tumble dryer

 Do not iron  Do not clean chemically

Close the Velcro fasteners in order to avoid damaging other items of clothing. Do not use fabric conditioner. Stretch back into shape and dry in the fresh air.

### **Guarantee**

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for

the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

### **Useful life/Lifetime of the product**

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

### **Duty of notification**

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA).

You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact).

### **Disposal**

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

### **Declaration of conformity**

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: [www.bort.com/conformity](http://www.bort.com/conformity)

Status: 10.2022

Medical device |  Single patient multiple use

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