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1 General instructions

1.1 Introduction

These operating instructions contain information on how to use the raised toilet seat safely and properly.

Read the operating instructions thoroughly prior to the use of the raised toilet seat and follow all instructions.

Keep all documents in a safe place.

1.2 Proper use

The raised toilet seat is used exclusively to raise the toilet seat and is therefore intended to be fitted to a toilet bowl.

Any other form of use is not permitted.

The raised toilet seat is only intended for adults. Do not use as a children's toilet seat.

Contraindications

No contraindications are known if the product is used properly.

1.3 Disposal

The product consists of recyclable materials.

Old appliances and packaging must be disposed of and recycled in accordance with the valid national regulations.

1.4 CE mark

The product complies with the medical products directive (EU) 2017/745.

1.5 Reuse

The product is suitable for reuse. The amount of reuse depends on the condition of the product in question.

Prior to reuse, follow the manufacturer's reconditioning and hygiene guidelines, and check the technical and functional safety of the product as well.

1.6 Service life

The product is designed for a service life of at least 3 years, provided that it is used in accordance with its intended use and that all cleaning and safety instructions are observed.

This service life can be exceeded if the product is carefully handled, cared for, and used.

2 Safety instructions

Do not overload the raised toilet seat (max. user weight: 200 kg).

In the case of a wall-mounted toilet bowl, do not exceed the permissible load weight of the toilet bowl.

Make sure that the raised toilet seat and buttocks are dry before going to the toilet.

Persons with reduced mobility should move onto the raised toilet seat from the front, not from the side.

Do not put weight on the raised toilet seat on one side only.

Do not use the raised toilet seat if it is damaged.

Ensure that the raised toilet seat is attached to the toilet bowl such that it cannot slip.

Do not sit on the raised toilet seat for extended periods.

Do not use raised toilet seat as a climbing assistance nor stand on it.

When using the raised toilet seat, keep a still posture and avoid excessive body movements. Otherwise, there is a risk of slipping.

In the following cases, only use the raised toilet seat when accompanied by an assistant: if you have a balance disorder or sensory disorder; if sitting independently is not possible; if the upper body cannot be held upright independently.

Sunlight or hot air can cause the surface of the product to become hot. Risk of burns!

Do not modify the product.

Check the product periodically for damage and safe assembly. Check that the clamping screws are sufficiently tightened.

Serious incidents relating to the product must be reported to the manufacturer and the competent authority.

3 Description

The raised toilet seat is available in the following versions:

- with cover (Fig. 1)
- without cover (Fig. 2)

3.1 Technical data

Refer to "Technical data" at the end of the operating instructions.

3.2 Label

The label (Fig. 6) is attached to the bottom side of the seat and contains the following information (exemplary illustration):

Sales or service address (1, Fig. 6), conformity marking (2, Fig. 6), note (observe operating instructions) (3, Fig. 6), maximum user weight (4, Fig. 6), period of production (5, Fig. 6), product identification number (6, Fig. 6), product name (7, Fig. 6), marking as a medical product (8, Fig. 6), batch number (9, Fig. 6), product type (10, Fig. 6), barcode (11, Fig. 6), item number (12, Fig. 6) and manufacturer (13, Fig. 6).

3.3 Attaching the side clamps

CAUTION Risk of catching. Attach the side clamps carefully.

1. Push the side clamps (2, Fig. 3) into the recess (1, Fig. 3) on the bottom edge of the seat until they touch the thread.

NOTICE Do not use a tool. Ensure that the thread of the seat and clamping screw are not damaged.

2. Carefully rotate the clamping screw (3, Fig. 3) clockwise, applying light pressure, until it engages in the thread and the side clamp is visibly pulled further into the seat.

3.4 Attaching the raised toilet seat

1. Fold the existing toilet seat (1, Fig. 4) upwards.
2. Place the raised toilet seat on the toilet bowl. The raised toilet seat must sit evenly and may not tilt.
3. Push the raised toilet seat backwards until the front end stop (2, Fig. 4) touches the toilet bowl.
4. Rotate the clamping screws (4, Fig. 4) clockwise manually until the side clamps (3, Fig. 4) touch the toilet bowl evenly. The raised toilet seat must sit centrally on the toilet bowl.

NOTICE Do not use a tool. Only tighten the clamping screws hand-tight. Do not over-tighten clamping screws. Otherwise, the plastic thread may be damaged.

5. Tighten the clamping screws (4, Fig. 4) manually, so that the raised toilet seat cannot slip.

3.5 Removing the raised toilet seat

1. Rotate the clamping screws (4, Fig. 4) anti-clockwise.
2. Remove the raised toilet seat.

3.6 Attaching the cover

1. Place cover (1, Fig. 5) on the seat (2, Fig. 5) such that the hooks (4, Fig. 5) in the cover each touch the brackets (3, Fig. 5).
2. Push on the rear edge of the cover until the hooks engage.

3.7 Removing the cover

1. Fold the cover around 90° upwards.
2. Close to the brackets, push the cover away from the seat until the hooks disengage from the brackets.

4 Maintenance and care

NOTICE Do not use any scouring agent. Otherwise, the plastic surfaces may be damaged.

The product can be cleaned with standard cleaning agents (without solvents and with a pH value of 5-9).

If necessary, or when using the product for more than one person, the surface can be disinfected with an off-the-shelf disinfectant (max. 70 % alcohol) **after cleaning**. Depending on the disinfectant used, the disinfectant will be sprayed on or applied with a cloth. Follow the disinfectant or detergent manufacturer's instructions.

When adhering to the specified cleaning and safety instructions, the product is maintenance-free.

After cleaning, dry off the raised toilet seat with a clean cloth.

Have any necessary repairs done by an authorised specialist workshop.

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