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99907 830 19338

## User Instructions

# Rotatable aluminium bath seat with armrests and backrest

Item No. 78062 300

**Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.**

The rotatable aluminium bath seat with armrests and backrest offers persons with a restricted ability to stand but who are still able to sit, support in carrying out daily hygiene in the bath. The cut-out in the seat facilitates intimate hygiene. The rotatable seat area makes it easier to enter and leave the bath seat and so increases safety.

### Safety Indications

- Users who weigh more than 130 kg must not use the rotatable bath seat!
- Use of the rotatable bath seat assumes the person doing so is able to sit down without aid. Do not use the bath seat if you are not able to control your balance when sitting down!
- For your own safety we recommend the aid of another person when entering and leaving!
- Only use the bath seat when it is technically in perfect condition!
- The bath seat is solely designed for the purpose of sitting in a bath!
- Only use the bath seat when you are sure it is securely fixed!
- Before using the seat, always make sure that it is securely positioned on the edge of the bath! For this to be the case at least 2/3rd of the width of the four rubber support flaps must lie on the edge of the bath!
- When entering and leaving the bath and when taking a bath, make sure that the bath seat is securely locked!
- After rotating the seat, always make sure that the locking lever has properly snapped back into position!
- Under no circumstances use the bath seat if one or more of the rubber support flaps is missing, there is a risk that it will tilt over!
- There is an increased risk of becoming caught in and squashed in the swivel area and at all the joints of the rotatable bath seat!
- Do not lean too far forward when sitting on the rotatable bath seat!
- Do not load just one of the armrests when sitting down or standing up!
- Do not make any structural changes to the bath seat which have not been authorised by the manufacturer!
- Please note that there is a risk of slipping when sitting down and standing up in a wet bath - in particular if you have used bath foam or soap!
- Children are only allowed to use the rotatable bath seat with armrests and backrest under the supervision of an adult!

### Scope of delivery

The materials supplied include: 1 rotatable bath seat, 2 bath rests incl. rubber buffers, 2 arm rests, 1 back rest, 1 copy of the instructions on use, 1 box for packaging and transport. Check the contents of the package.

If there are obvious defects or if parts are missing please get in touch with your specialist retailer.

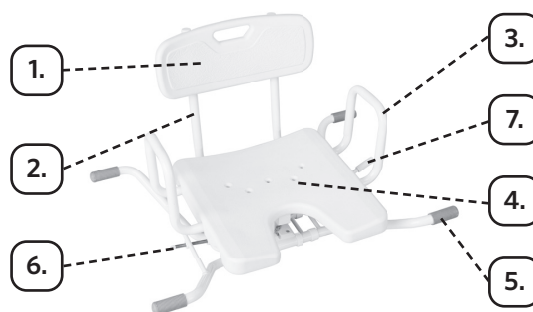
### Assembly and use

Place the seat on a table with the underneath facing upwards. Push the right-hand bath rest into the tubes to the right of the seat. You can choose between two positions to adjust the width. Select the one that fits the width of your bath. Secure the width setting with the aid of the two E clips. Repeat the process on the left hand side. Ensure that you select the same width setting for both the right and left hand sides. Fit the armrests into the tubes to the sides of the seat by pressing the spring pins in. Finally assemble the backrest by pressing the spring pins in and fitting it into the tubes behind the seat.

A clearly audible click indicates that the arm rests and back rest have been positioned securely.

### Components:

1. Backrest
2. Frame
3. Armrests
4. Seat with drain holes
5. Support flaps
6. Locking screws
7. Locking lever

### Use

1. The bath seat has two width settings. You can select a width of either 78 or 84.5 cm. Place the rotatable bath seat in the middle of the bath so that at least two-thirds of the support flaps safely and firmly sit on the edges of the bath. Secure the width setting with the aid of the E clips supplied as described under »Assembly«.
2. Unscrew the locking screws until the plastic heads are in contact with the inside of the bath. This provides additional safety against slipping. The locking screws can be adjusted between 51 and 67 cm.
3. The bath seat can be rotated through 360° and can be locked in four different positions. Before entering the bath, rotate the seat to the enter position. Loosen the lock by pulling the locking lever upwards – this pulls it out of its position.
4. Turn the seat towards you and then push the locking lever down again so that it can snap into the corresponding opening. Now the seat can no longer be rotated. Only sit down on the seat when it has been locked in position!
5. As soon as you are sitting comfortably, pull up the locking lever again so that from the sitting position you can turn in the desired direction. When you are in the position you want, snap the locking lever back into position.
6. When getting out of the seat make sure that it is firmly locked, so that you are able to get out of it safely. Where possible arrange to have another person present to help you.

### Cleaning/Disinfection

The rotatable bath seat with armrests and backrest should only be used on intact skin. A cleaning agent that is gentler to material should be used for manual cleaning. Only DGHM-/VAH-listed material-friendly surface disinfectant should be used. Please see the relevant manufacturer's instructions for a selection of suitable cleaning agents and disinfectants and their use.

### Change of patient

The product can be re-used. It must be cleaned and disinfected according to instructions. It must be inspected by specialist shops for any faults and then authorised before it can be used again. Please order our hygiene and maintenance guidelines.

### Material

- Seat/back rest: PE
- Frame and locking lever: powder coated aluminium tube
- Support flaps: rubber

### Disposal

Do not dispose of the bath seat in the normal household rubbish. Ask your local recycling authority how to dispose of it properly and in an environmentally friendly way.

### Technical data\*

- Dimensions of the seat: 40 x 41 cm (width x depth)
- Height of backrest (from the seat to the upper edge of the backrest): 35 cm
- Height of the armrests (from the centre of the seat to the upper edge of the armrests): 18 cm
- Overall height: 51 cm
- Overall width: 78–84,5 cm
- Overall depth: 53,5 cm
- Distance between the armrests: 50,5 cm
- Maximum weight of user: 130 kg
- Weight: 4,4 kg

\* Slight deviations may occur.

This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at [info@rusksa.de](mailto:info@rusksa.de) as well as the competent authority, the Medicines & Healthcare products Regulatory Agency MHRA, [www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency](http://www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency).

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Fabrikant: Ludwig Bertram GmbH, Im Torfstich 7, 30916 Isernhagen, Duitsland, [www.russka.de](http://www.russka.de)