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# Badewannensitz

Art.-Nr. 78062 100 Aluminium

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Art.-Nr. 78062 200 Aluminium

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# Bath seat with backrest

Item-No. 78062 200 Aluminium

**Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.**

The bath seat / bath seat with backrest offer persons with a restricted ability to stand but who are still able to sit, support in carrying out daily hygiene in the bath. The cut-out in the seat facilitates intimate hygiene. The bath increases safety.

### Safety Indications

- Users who weigh more than 130 kg must not use the bath seat!
- Use of the bath seat assumes the person doing so is able to sit down without aid. Do not use the bath seat if you are not able to control your balance when sitting down!
- For your own safety we recommend the aid of another person when entering and leaving!
- Only use the bath seat when it is technically in perfect condition!
- The bath seat is solely designed for the purpose of sitting in a bath!
- Only use the bath seat when you are sure it is securely fixed!
- Before using the seat, always make sure that it is securely positioned on the edge of the bath!  
For this to be the case at least 2/3rd of the width of the four rubber support flaps must lie on the edge of the bath!
- Under no circumstances use the bath seat if one or more of the rubber support flaps is missing, there is a risk that it will tilt over!
- When adjusting the width and when reaching into the gap between the frame and the seat there is a risk that fingers and hands may become trapped.
- Do not lean too far forward when sitting on the bath seat!
- Do not load just one of the armrests when sitting down or standing up!
- Do not make any structural changes to the bath seat which have not been authorised by the manufacturer!
- Please note that there is a risk of slipping when sitting down and standing up in a wet bath - in particular if you have used bath foam or soap!
- Children are only allowed to use the bath seat with armrests and backrest under the supervision of an adult!

### Scope of delivery

The materials supplied include:

78062 100: 1 bath seat with frame, 2 sides,

78062 200: 1 bath seat with frame, 2 sides, 1 backrest, 1 bath seat or 1 bath seat with armrests and backrest,  
1 copy of the instructions on use, 1 box for packaging and transport.


Check the contents of the package. If there are obvious defects or if parts are missing please get in touch with your specialist retailer.

## Assembly/Use

The bath seat/bath seat with armrests and backrest is delivered pre-assembled.

### Assembly 78062 100

Push the seat onto the frame until you feel it engage with an audible click. (ill. 1 and 2)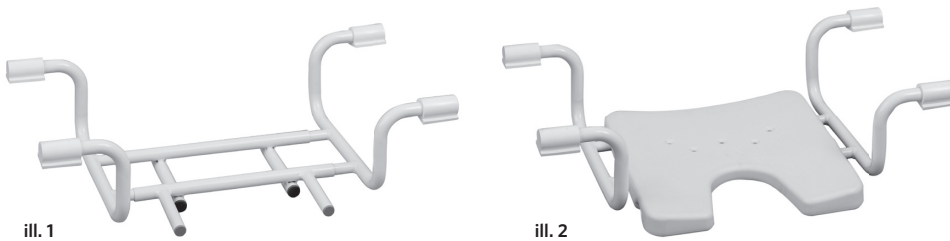


### Montage 78062 200

Place the seat down in front of you with the underneath facing up and the commode cut-out facing away from you. Push the tubes of the backrest into the tubes on the seat frame until you feel the spring pins engage with an audible click. (ill. 3)


To assemble the side sections, push the side sections into the tubes on the seat so that the spring pins can engage in the openings provided for them. (ill. 4, 5)

To adjust the width please read the instructions below.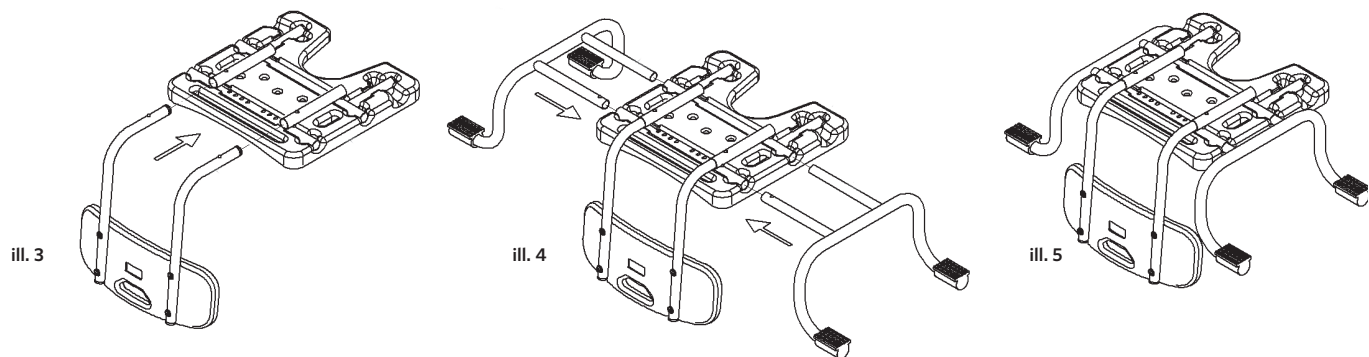

### Components:

1. Support flaps
2. Seat with drain holes
3. backrest

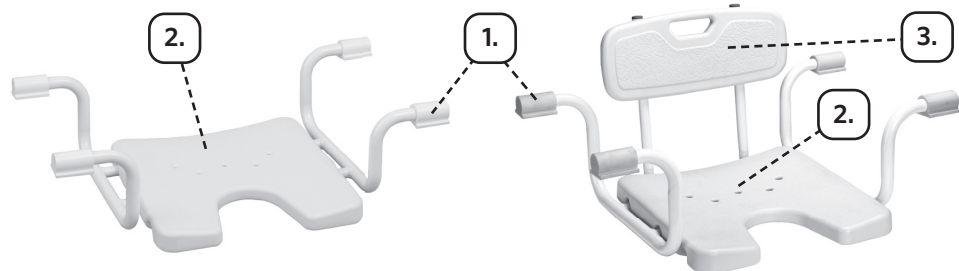


### Adjusting the width of the bath seat 78062 100, 78062 200

The width of the bath seat can be adjusted. This adjustment is made at the underside of the seat.

1. Insert the spring pins and pull the sides. (ill. 7)
2. Place the bath seat in the bath so that you can decide on the optimum width. Pull open the two sides of the supporting frame so that the support flaps lie fully (and by at least 2/3rd) on the edge of the bath.
3. Make sure you pull out the left and right sides of the frame to the same extent! (ill. 9)
4. Engage the spring pins at the desired position. The width of the bath seat is only safely adjusted when the pins are visibly and noticeably protruding from the holes and accidental displacement of the tubes is prevented. (ill. 8)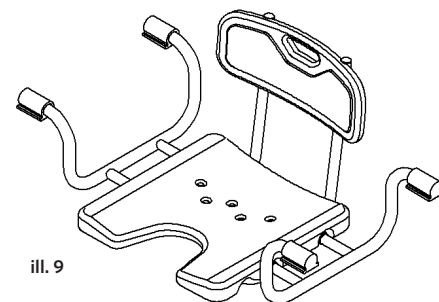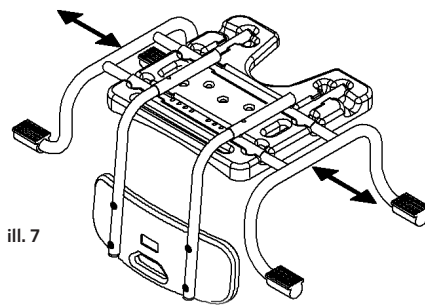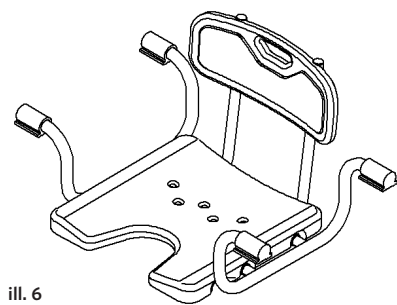

### Placing the bath seat in the bath

1. You can use the bath seat with every commercially available bath of normal dimensions.
2. The rubber support flaps prevent slipping and also protect the edge of the bath from scratches. Before using the seat, always check that these flaps lie fully (and by at least 2/3rd) on the edge of the bath!

### Cleaning/Desinfection

Clean the bath seat/bath seat with backrest using a mild household cleaner. Do not use any corrosive or abrasive cleaning agents. To wipe down and disinfect the surfaces use a standard disinfectant with an alcohol content of a maximum of 75%. Then rinse with water.

### Change of patient

The product is suitable for further use. Clean and disinfect it in accordance with the instructions. It should also be checked by a specialist for damage, and then approved for further use.

### Material

- Seat/backrest: PE
- Frame and locking lever: powder coated aluminium tube
- Support flaps: rubber

### Disposal

Ask about your local disposal regulations.

### Technical Details\*

Item	Dimension Seat	Dimension backrest	Total height	Total width	Total depth	Max. user weight	Weight
78062 100	40,5 x 40,5 cm	—	23 cm	73-85 cm	44 cm	130 kg	2,5 kg
78062 200	40 x 40,5 cm	From seat to upper edge of the backrest: 38 cm	49 cm	73-85 cm	46 cm	130 kg	3,1 kg

\* Slight deviations may occur.

This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at [info@rusksa.de](mailto:info@rusksa.de) as well as the competent authority, the Medicines & Healthcare products Regulatory Agency MHRA, [www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency](http://www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency).

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Fabrikant: Ludwig Bertram GmbH, Im Torfstich 7, 30916 Isernhagen, Duitsland, [www.russka.de](http://www.russka.de)