# NAVIGATOR Air

## Outdoor rollator with forearm support

User's manual (EN)

## Der Outdoor Arthritis Rollator

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# ENGLISH

## 1 General Information

Dear Customer,

This user manual contains a description of the medical device and important guidelines to ensure correct and safe use of the product. It is important to read this manual carefully prior to use. It is especially important to read the safety requirements and follow these.

**REHASENSE** continuously develops its products and reserves the right to change the specifications and functions of products without notice.

If you have any queries please contact your dealer or **Rehasense** directly. Contact information is located on the last page of this manual.

## 2 Intended usage, Indications & Contra indications

**Intended usage:** The rollator has been designed as an assistance walking tool for people who have mobility problems with standing, walking or seating. Thanks to the rollator those persons can stand, walk and rest easily and they can fulfil without others assistance most of their daily tasks and routines. Any other usage is prohibited. The **Navigator Air** rollator is designed for both outdoor and indoor use.

**Indications:** The device is specifically indicated for individuals who, (because of a wide variety of possible health problems) require additional stable support during walking. It is specially designed for users with arthritis and/or limited strength in their arms and hands who cannot support on their wrists but still can use their forearms for support.

**Contraindications:** The device must not be used by persons who have serious problems with body balance or who's muscle strength is too low to be able to stand and walk, even with extra support of a rollator. The need and possibility of usage of a rollator by user should be always estimated and indicated by a physician or physical therapist.

## 3 Delivery & general product description

The rollator is supplied in boxes and come folded with disassembled forearm supports, for easier transportation.

The contents are:

- 1 complete, folded rollator with:
  - Forearm support assembly left + right
  - Handgrip assembly (handgrip and brake lever left + right)
  - Stabilisation strap between the forearm supports
  - 1 mounting kit with 4 bolts and 1 Allen key
  - 1 small net bag to put between the push handles.
- 1 user's manual

The **Navigator Air** is a four wheel rollator with high forearm supports and large wheels. The very stable and comfortable forearm supports are height adjustable for users to set it exactly according to their needs and as an option it can be featured with horizontal adjustment. The **Navigator Air** is made of aluminium and has a cross foldable frame, so it is light and easy to store or to transport if needed. It is also equipped with a fabric seat which allows the user to rest and with a shopping bag to carry small items.

## **4 Security requirements**

Make sure that this user's manual is read by all people who use the device. The manufacturer does not take any responsibility for damages and/or injuries caused by the fact that the user's manual has not been followed.

### **Limitations for usage**


- The device must not be loaded with more than 200 kg.
- The device's bag has load limitation of 5 kg.
- The device is not intended to be self-propelled while seated
- The brakes **MUST** be in the locked position **BEFORE** using the seat.

### **Avoiding accidents**

- Use the device only for the purpose it is intended for.
- Use the device only in good technical condition.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device, the user must take care not to trap their fingers or other body parts between two moving elements of the device
- The device must be only used on stable surfaces.
- All wheels **MUST** be in contact with the ground at **ALL TIMES** during use. This will ensure the rollator is properly balanced and should avoid accidents.
- When using the rollator in a stationary position the hand brakes **MUST** be locked.

IF ANY SERIOUS INCIDENT OCCURES IN RELATION TO THE DEVICE PLEASE CONTACT YOUR DEALER OR REHASENSE DIRECTLY (contact details at the end of the user manual). The immediate actions should be taken according to the requirements of Regulation of the European Parliament and of the EU Council 2017/745 on medical devices of April 5, 2017.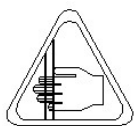

Mind your fingers

**Warning:** Fingers or other body parts of a user and/or assistant can be trapped between the device's parts during folding, unfolding or adjusting of the device. Be extremely careful when adjusting the device. Observe the dimensions of gaps between the device's elements to avoid trapping your fingers or other body parts.

## 5 Product details

## 6 Usage

Using the above image, the device can be prepared for use. However, we encourage you to follow the instructions below step by step.

### Initial Check

- In the first instance take the device out of the box and check (using the image above) that nothing is missing.
- If you discover at this stage that some part is missing or damaged do not continue to assemble the product. Contact your supplier of the product right away.

### Assembly of the rollator

- Mount the left and the right forearm supports (Picture 1) on the corresponding tubes in combination with the clamps of the stabilization strap (Picture 2) using the four screws and the Allen key from the mounting kit.
- Insert the push handles with the brake levers (Picture 3) into the forearm support assembly and preliminary secure with the clamping lever (2)
- Connect both sides of the backrest with the buckle in the middle (4)
- Close the buckle of the stabilizing strap (3) between the forearm supports. With the Velcro material the length of the strap should be adjusted in such a way that there is a light pulling force.

#### Opening of the rollator

- Push the red button which is locking the seating tubes (See the image shown on the right side).
- Hold both seat tubes and simultaneously push them down until you hear a “click” sound of the locking mechanism of the frame.
- Try to pull the seat tubes back up to fold the frame. If this is not possible the frame is opened and secured correctly.

*After unfolding or assembling the rollator always ensure that the seat support is securely locked by the pulling strip mechanism for securing the lock and that the frame does not move anymore.*

- *Always ensure that both arm support inner tubes are securely locked in the main frame tube with the two wing knobs to prevent the accidental sliding in of the inner tubes when forced.*
- *Always test that both armrests are set on the same height and that they are properly fastened on top of the inner tube. It is not allowed to use a defective device.*
- *Always make sure that after everything has been secured, nothing can move.*
- *Always test to see that the rollator and attachments are properly and safely locked in place BEFORE using.*
- *Failure to follow these warnings can result in personal injury.*

### **Folding**

- Disconnect the stabilization strap (3)
- Disconnect the backrest in the middle (4)
- Pull at the pulling strap on the seat to fold the rollator.
- Fold the rollator until the red button locks

*- Before the rollator is folded and locked, one of the forearm supports must be set to a minimum of three steps higher. See pictures 2 and 3.*

### **Adjusting the height of the forearm supports**

- Disconnect the stabilizing straps
- Disconnect the backrest (4) in the middle - Release the height adjustment's locking knob
- Make adjustment of the forearm supports to your desired height.
- Tighten the knob.


- Adjust the backrest clamps (5) to the appropriate height of the backrest and tighten the bolts
- Reconnect both sides of the backrest (4)
- Reconnect the stabilizing strap (3), to ensure full rigidity and stability of the structure.

- Both arm pads should be adjusted to the same height.
- If the arm pads are correctly adjusted, the user must stand upright.
- Always lock the stabilizing strap before applying load on the arm pads.
- Failure to follow these warnings can result in personal injury.


#### Adjusting the length and angle of the push handles

- Release the handle's length locking lever (2)
- Make adjustments to the desired length and angle, so it is easy and comfortable to hold the handle.
- Tighten the lever.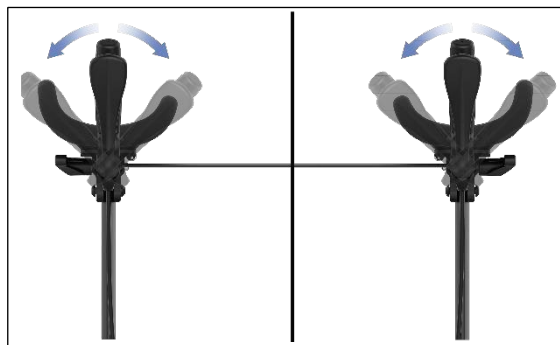

#### Adjusting the angular rotation of the forearm support (*optional*)

- Release the forearm support's locking knob (1)
- Make adjustment to the desired angle to fit your personal comfort - There are 3 possible angles to set: 0°, 22,5° and 45°
- Tighten the knob.

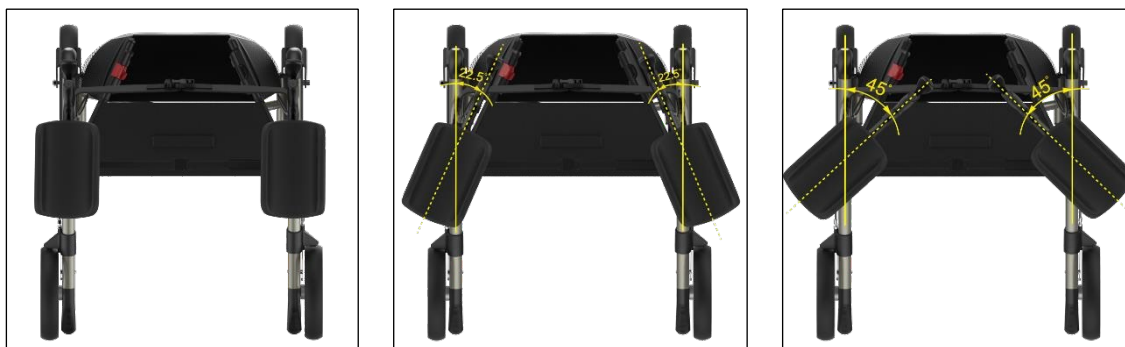- The small net bag can only be used for positions 0 ° and 22,5 °.

### Using the hand brakes

- TO STOP THE ROLLATOR pull the brake levers towards the hand grips.
- To stay stationary: keep the brake levers squeezed.
- *The braking force can be increased by squeezing the brake levers more firmly*
- TO START MOVING release the brake handles. *They will move back automatically and the brakes will be released.*

- *You should always engage the breaks when you don't plan to move the device.*
- *Always engage the parking brakes before sitting on the rollator*
- *Never leave the device on slopes without the breaks engaged!*
- *Never sit too much to the rear. Do not sit (partly) above the shopping bag.*
- *Never lean backwards when seated on the rollator*

### Locking/Unlocking the hand brakes

- To lock the brakes push the brake handles forward until you can hear the "click" sound.
- To unlock the brakes pull the brake handles backwards until you can feel them release.


- *When using the rollator in a stationary position, the hand brakes MUST always be locked.*

### Tilt aid

Your walker is equipped with a tilt aid on both sides for obstacles such as door treads, curbs, etc.

- Pull the brakes, put your toes on the step, push it down and pull the handles towards you so that the front wheels come off the ground.
- Keep your heel on the floor to keep your balance.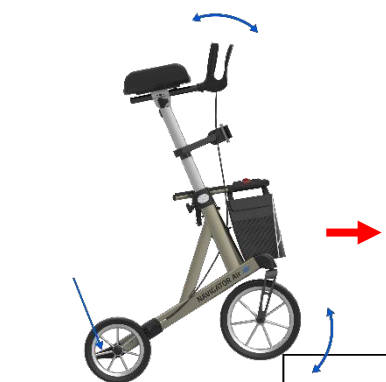

### To sit on the rollator

To rest a while you can sit on the seat of the rollator.

- First put both the parking brakes on.
- Then you can turn around to take a seat.
- Sit between both handles in active seat (upright).
- You can lean on the handles with the forearms.
- Do not sit too far back.

- *Never sit too far back, not (partially) above the shopping bag.*
- *Never lean back.*
- *Never sit on the walker without the parking brake on.*

### Backrest

Navigator Air Includes a two parted height adjustable backrest.

- It has a fixed length for safety reasons.
- Standard featured from the factory, height adjustable for individual needs.
- First adjust the height of the forearm supports and then adjust the height of the backrest about 20 cm above the seat for the best comfort and support.



### Steering and turning of the rollator

- Due to the low weight and easy swiveling front wheels, the walker is very easy to maneuver forwards, sideways or backwards and can be turned on the spot.
- Simply place your arms on the forearm supports, hold the push handles and push the walker in the desired direction.

### Reflectors

For safety reasons, the rollator is provided with a total of 9 reflectors to be better seen in the dark: 2 on the side, 2 on the front fork holder, 3 reflective seams on the shopping bag and 2 stripes at the back of the inner tubes.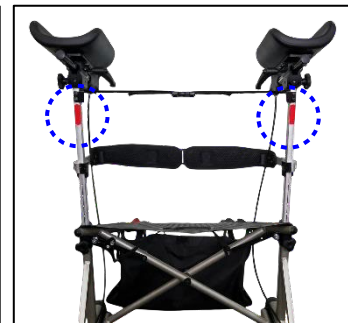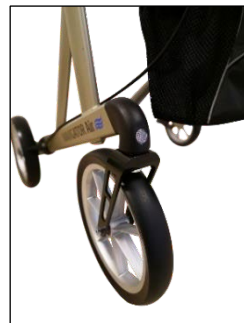

## 7 Cleaning & Disinfection

### General:

- Clean the device regularly and keep it clean.
- Lighter dirt and dust can be removed with a wet cloth.
- Larger parts of dirt can be removed with a non-corroding, mild household detergent (available in most stores) and warm water.
- Do not use harsh abrasives or bleach based cleaners, as this may cause damage or discolouration. Dry parts with a soft cloth.

Wheels:

- Clean wheels with warm water and a mild cleanser mild household detergent (available in most stores). Dry with a clean rag.
- Lubricate with multi-purpose oil periodically or if wheels begin to squeak.


Disinfection:

For disinfection please contact your dealer to be sure is made by professional personnel.

However for individual use we recommend to use the generally available disinfectants without chlorine and phenol. The manufacturer is not responsible for the damage caused by wrong usage of disinfectants.

## 8 Label Example

The product label is located on the inside of the side frame.

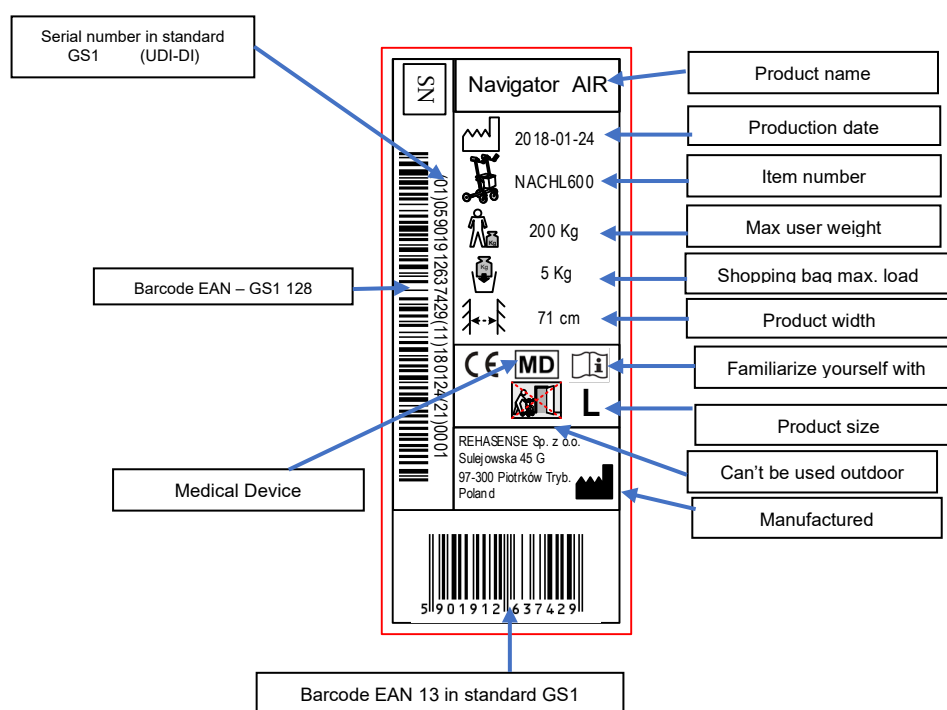## 9 Technical data

<b>NAVIGATOR Air – Outdoor rollator with forearm support</b>	
Frame size	Large
Recommended user height	140-185 cm
Max. users weight	200 kg
Total weight (without shopping bag)	12 kg
Weight of shopping bag	0,4 kg
Height adjustable forearm supports	87-110 cm (9 steps)
Forearm supports with horizontal rotating	22,5° or 45° inward
Distance between forearm supports	40 cm
Distance between the handgrips (straight position)	56 cm
Total height, unfolded (min-max)	96-120 cm (1-9 steps)

Length	77 cm
Width opened	71 cm
Width folded	31 cm
Height folded (minimum)	99 cm
Seat width	53 cm
Seat height	62 cm
Seat depth	18 cm
Max. load of the bag	5 kg
Brakes	2 x maneuver hand brakes with parking function
Wheels (diameter x width)	Front: 285 x 44 mm – Black SOFT PU tire Rear: 235 x 44 mm – Black SOFT PU tire
Turning radius	96 cm

Side frame tube diameter	50 x 25 mm (triangular)
Inner tube diameter of the forearm supports	40 x 16 mm (triangular)
<i>Used materials and surface treatment:</i>	
- Side frame	Aluminium (powder coated)
- Inner tube	Aluminium (anodized)
- Cross	Aluminium (powder coated)
- Wheels	PA + 15% GF and PU tire
- Forearm support	PU
- Hand grips	Soft TPE PA
- Brake handles	+ 30% GF
- Seat	PVC coated / Polyester
- Bag	Polyester

## **10 Transport description**

The rollator is an easily foldable product convenient for transportation. If it is necessary to transport it, we recommend to fold it as stated in the description above. The rollator should be transported in a way that it is stable and there is no risk of sudden and dangerous movements of the transported device.

## **11 Maintenance**

The rollator should be inspected.

- Ensure that every part and accessory of the rollator is secure at ALL times.
- Check all screws for tightness. If not, tighten if necessary.
- Replace any broken, damaged or worn items immediately.
- Check the braking function of both brakes and that the handbrakes perform well.

In case of any visible damages ask your local dealer for diagnostics and maintenance.

We recommend to ask your dealer to carry out inspection minimum once per 12 months.

## **12 Re-use**

This medical device can be reused by another user if the original user of the product doesn't need it anymore. Before reuse the device must be carefully inspected by authorized technical staff at the dealer.

Following elements should be checked:

- rigidity of the structure,
- tightness of bolts and nuts,

- condition of fabric elements,
- condition of wheels and ball bearings,
- efficiency and condition of brakes.

If any of above elements are torn or broken, they should be replaced with new parts. It is specially recommended to install new wheels for every new user of the device. All lose bolts and nuts must be tight. The product must be precisely cleaned and disinfected.

**WARNING!** It is forbidden to re-use the rollator when the frame is bent or broken.

Every new user must receive the device with this user's manual attached.

### **13 How to deposit the product**

The device should not be deposited with normal garbage. It should be delivered to the deposit station for reuse or recycling.

### **14 Warranty**

The guarantee is not valid by mis-intended use of the product.

The guarantee excludes damage caused by incorrect operation, accidents and/or through normal wear and tear. Any modifications of product will cause loss of guarantee. Mechanical damages are not covered by this warranty.

**For repairs during the guarantee period, please contact your local dealer**

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